



Journey S

Parts Breakdown
*For models: 4205BK, 4205BL, 4205PL, 4205RD
Serial # Starting "HF"*



CARE • QUALITY • INNOVATION




THE NOVA PROMISE

Revised on January 7, 2019

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

Thank you!




Figure 1

Hand Brake Mechanism

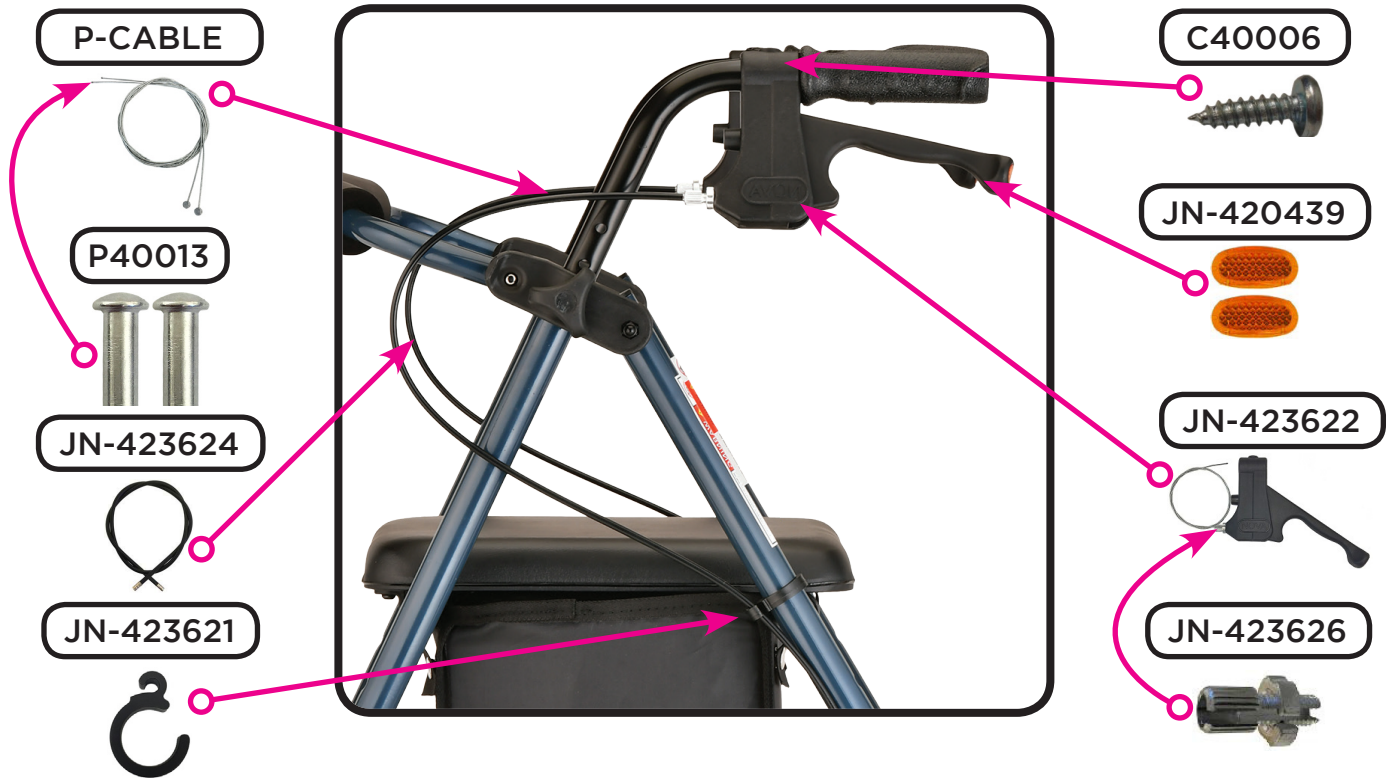

Upper Assembly Components

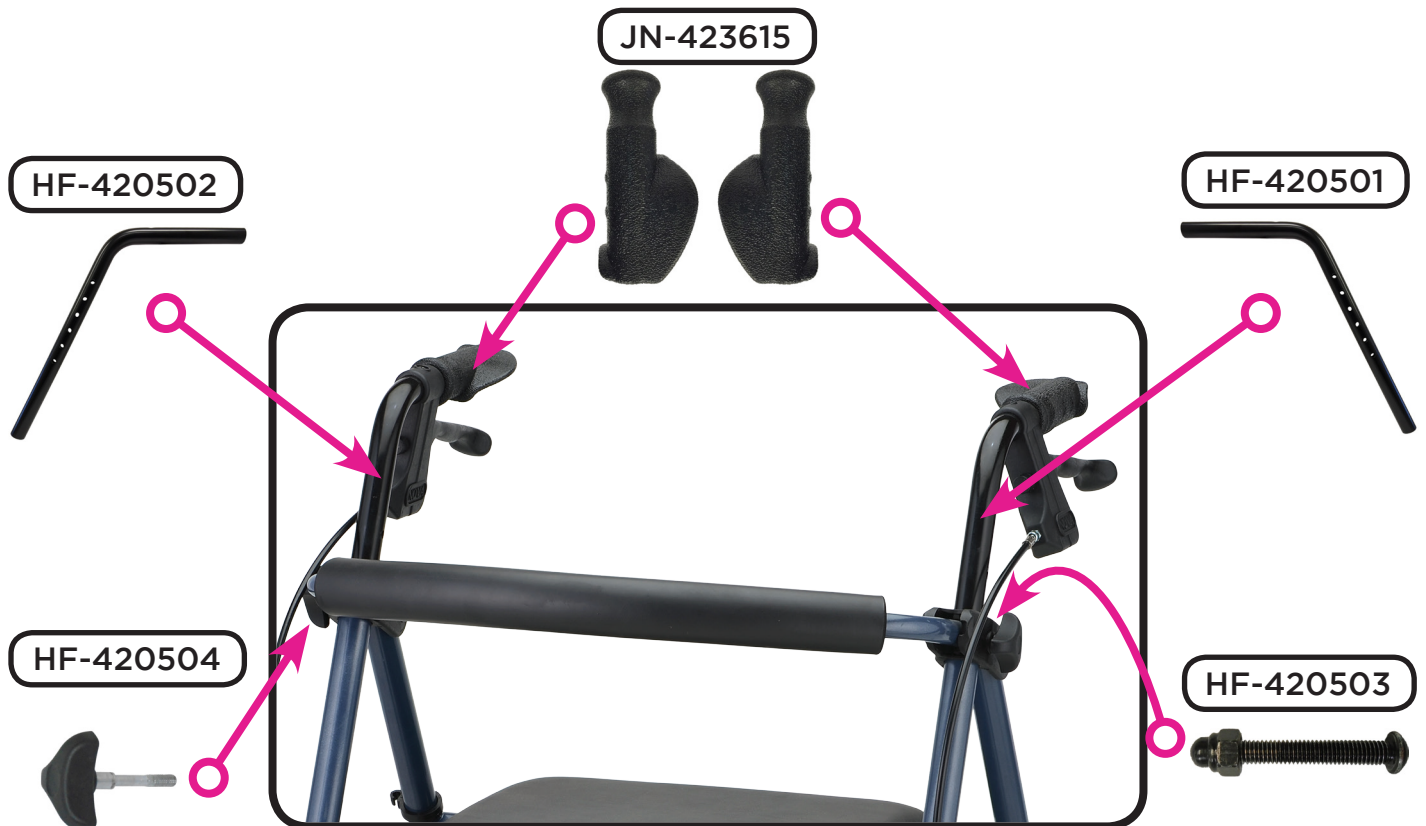

Seat & Backrest



Lower Assembly Components



Wheel & Lower Assembly Components

JN-423911



HF-420529



HF-420517

HF-420518

HF-420519

HF-420520

VT-33207



HF-420515



FRONT

JN-423630



HF-420513



REAR

HF-420521
HF-420523
HF-420525
HF-420527



HF-420516



HF-420514






Item Number 4205

JOURNEY S

SERIAL # BEGINNING WITH "HF"

ITEM NUMBER	DESCRIPTION	UOM
HF-420515	AXLE FRONT	EACH
HF-420516	AXLE REAR	EACH
HF-420508	BACKREST BLACK	EACH
HF-420509	BACKREST BLUE	EACH
HF-420510	BACKREST PURPLE	EACH
HF-420511	BACKREST RED	EACH
JN-423613	BAG	EACH
JN-423630	BEARING & SPACER (SET OF 4) FOR WHEEL	PAIR
VT-33207	BEARING FOR SWIVEL FRONT FORK	EACH
P-CABLE	BRAKE CABLE (PAIR)	PAIR
JN-423621	BRAKE CABLE HOLDER	EACH
HF-420514	BRAKE SHOE (SET) (INCLUDES HARDWARE TO ATTACH TO FRAME & SPRING)	SET
HF-420506	CLAMP AND SCREW (SET) FOR SEAT	SET
JN-423624	COVER FOR CABLE	EACH
P40013	CRIMP FOR STEEL CABLE (PAIR)	PAIR
HF-420512	FOLDING MECHANISM (INCLUDES ALL ATTACHMENT HARDWARE)	EACH
HF-420529	FORK FRONT ASSEMBLY (INCLUDES STEM, BEARINGS, & SPACER)	EACH
JN-423615	GRIPS (PAIR)	PAIR
JN-423622	HAND BRAKE WITH CABLE	EACH
HF-420501	HANDLE LEFT	EACH
HF-420502	HANDLE RIGHT	EACH
HF-420505	HINGE LEFT FOR A-FRAME AND BACKREST	EACH
HF-420530	HINGE RIGHT FOR A-FRAME AND BACKREST	EACH
HF-420504	KNOB HEIGHT ADJUSTMENT	EACH
HF-420517	LEG FRONT BLACK	EACH
HF-420518	LEG FRONT BLUE	EACH
HF-420519	LEG FRONT PURPLE	EACH
HF-420520	LEG FRONT RED	EACH
HF-420521	LEG REAR LEFT BLACK	EACH
HF-420523	LEG REAR LEFT BLUE	EACH
HF-420525	LEG REAR LEFT PURPLE	EACH
HF-420527	LEG REAR LEFT RED	EACH
JN-420439	REFLECTOR (PAIR) FOR HAND BRAKE	PAIR
HF-420503	SCREW FOR A-FRAME	EACH

	Item Number 4205	
	JOURNEY S	
	SERIAL # BEGINNING WITH "HF"	
ITEM NUMBER	DESCRIPTION	UOM
C40006	SCREW TO ATTACH BRAKE TO HANDLE	EACH
HF-420507	SEAT PADDED	EACH
JN-423911	STEM SCREW FOR FRONT FORK	EACH
JN-423626	TENSION SCREW	EACH
HF-420513	WHEEL 6" (INCLUDES BEARINGS & SPACER)	EACH

CONTACT US!
We're here to help



info@novajoy.com



1 800 557 6682

novajoy.com

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE