



# Mighty Mack

Parts Breakdown  
*For models: 4216BL, 4216RD  
Serial # Starting "NT"*



CARE • QUALITY • INNOVATION




THE NOVA PROMISE

*Revised on March 6, 2019*

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

**Thank you!**




**Figure 1**


## Hand Brake Mechanism

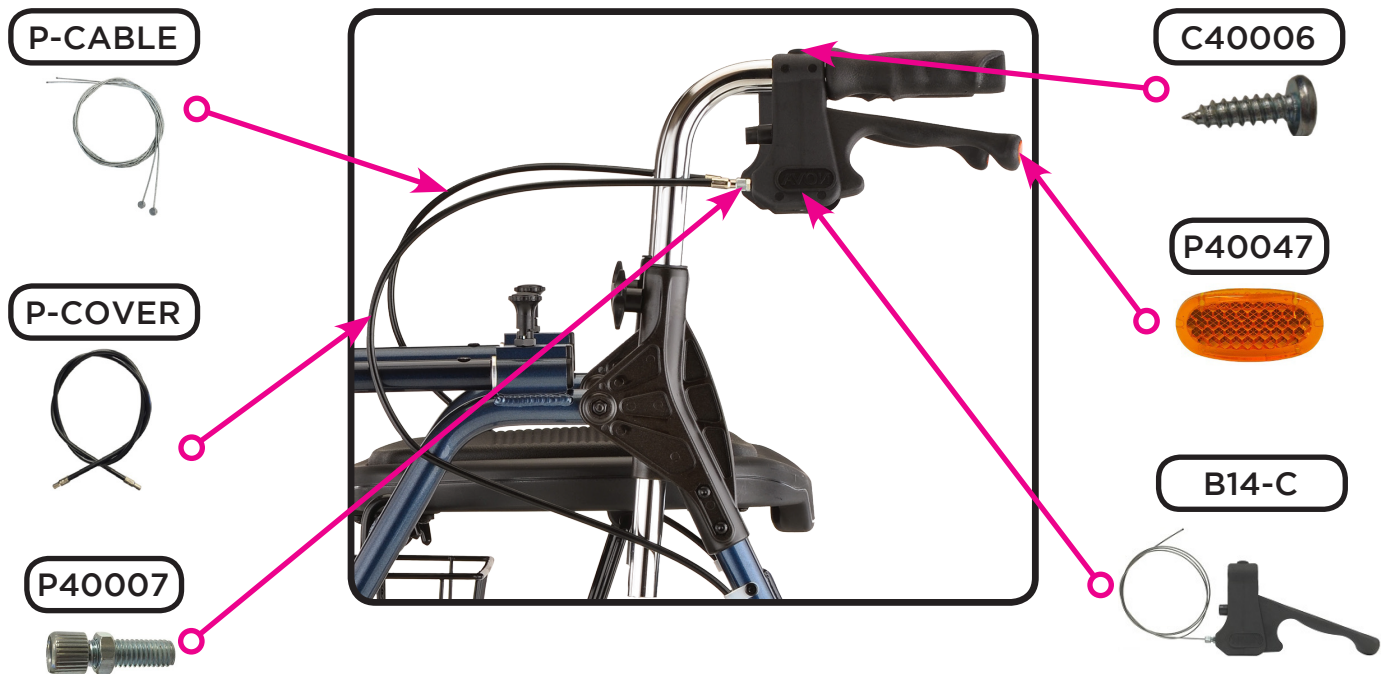

## Upper Assembly Components

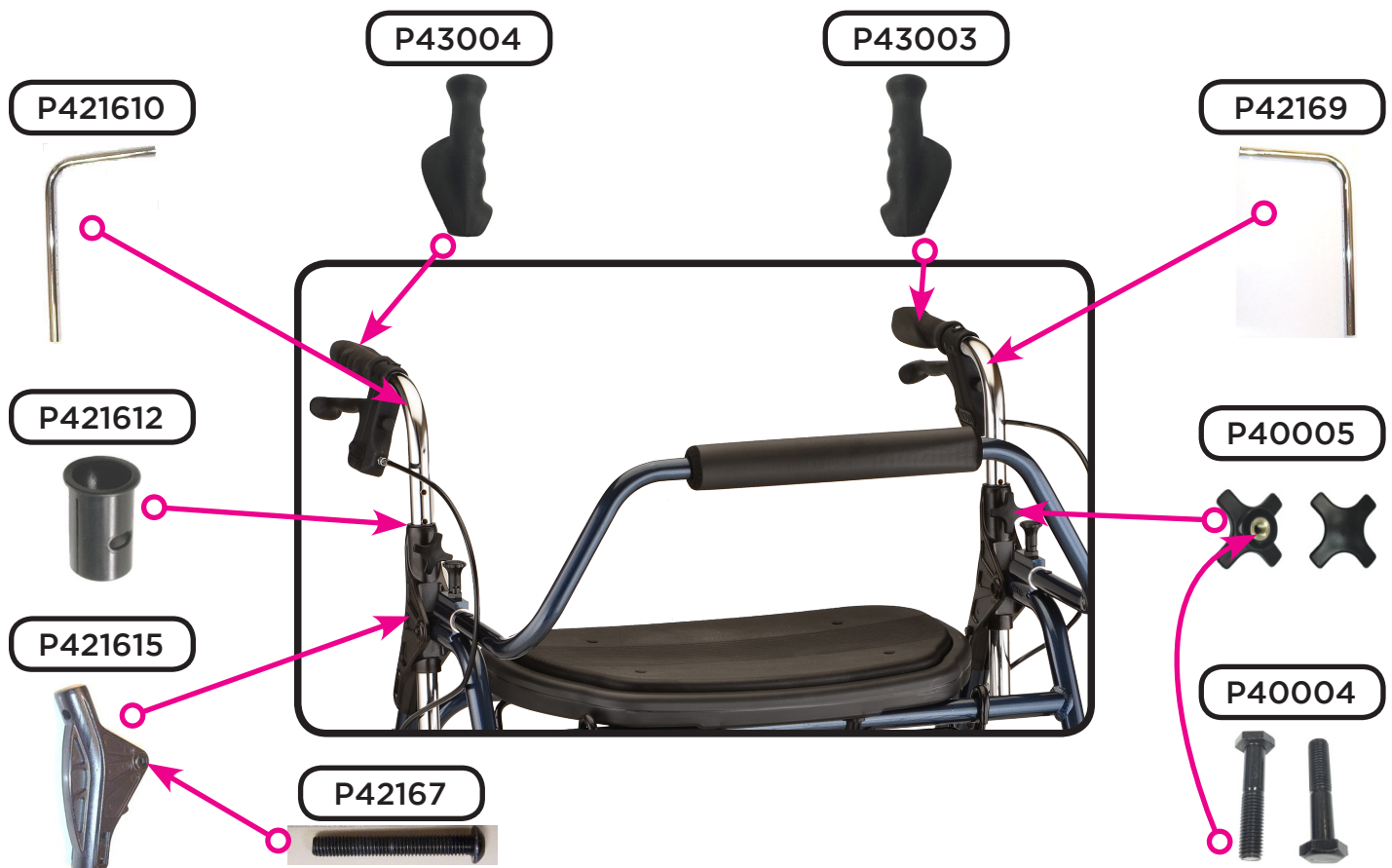

## Seat & Backrest

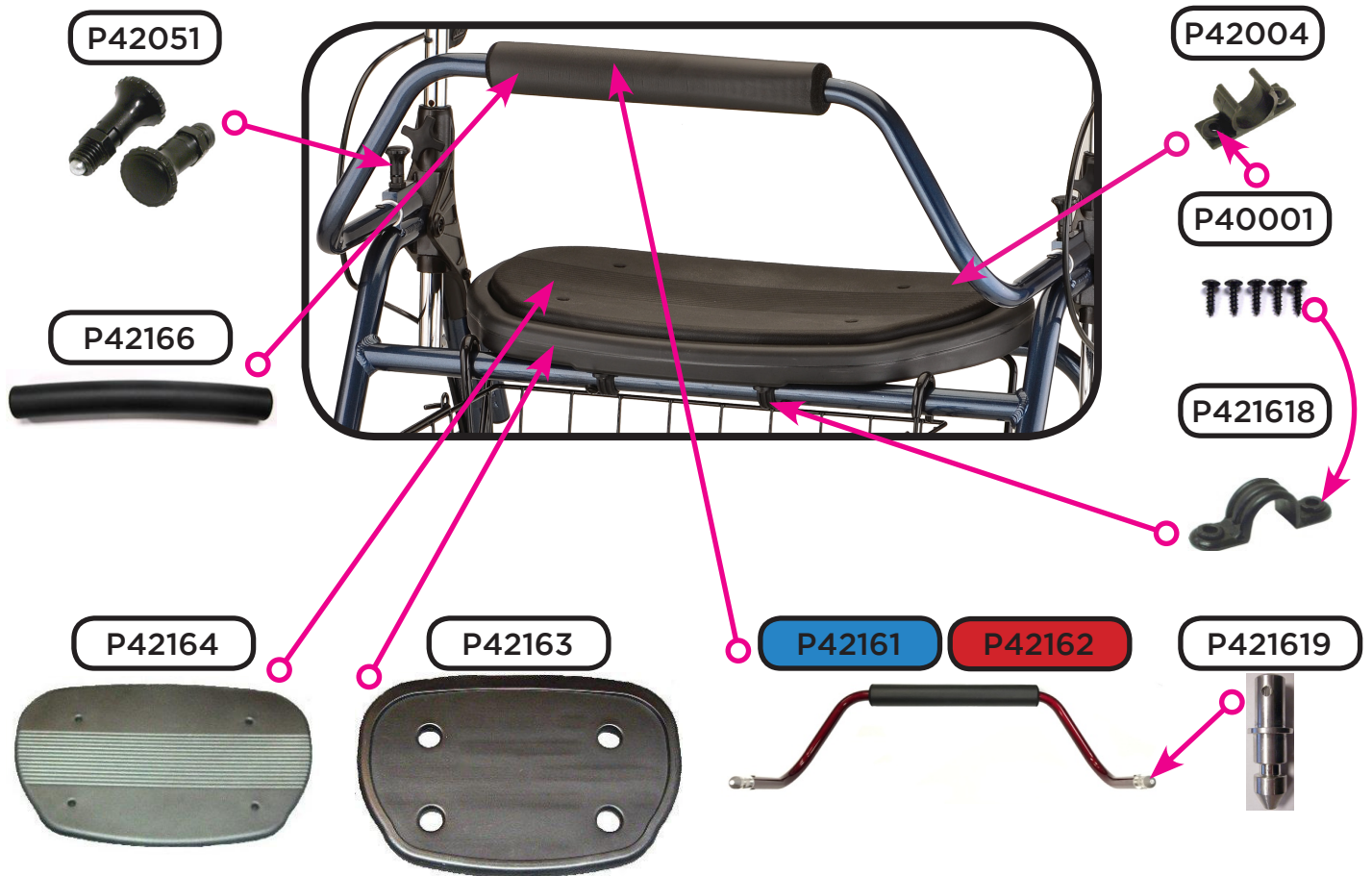


## Folding Mechanism & Accessories





## Wheel & Lower Assembly Components

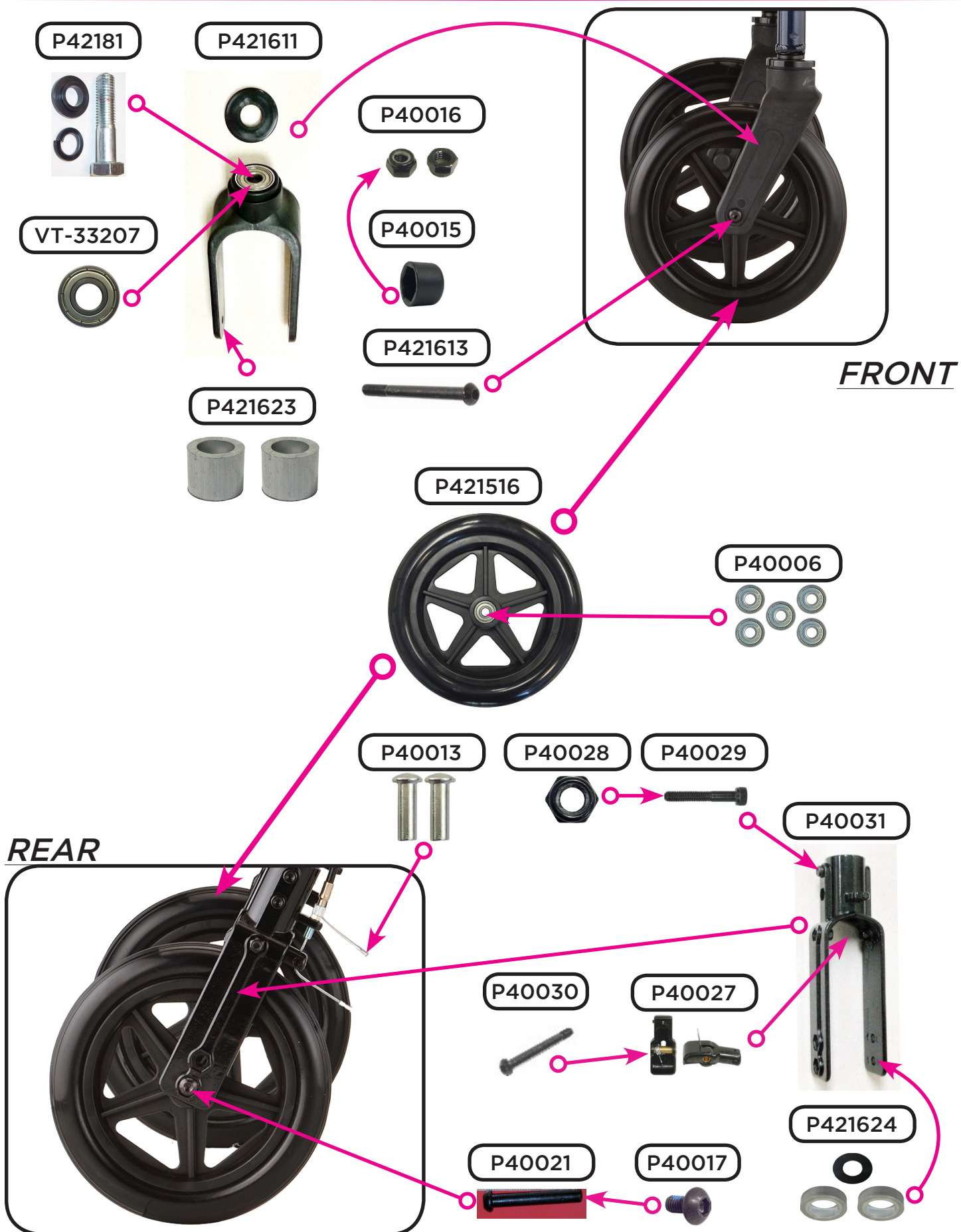




Item Number 4216

## Mighty Mack


SERIAL # BEGINNING WITH "NT"

ITEM NUMBER	DESCRIPTION	UOM
P42004	ANTI-RATTLE SEAT CLAMP FOR PLASTIC SEAT	PAIR
P40021	AXLE FEMALE FOR REAR WHEEL	EACH
P421613	AXLE FOR FRONT WHEEL	EACH
P40017	AXLE REAR WHEEL - MALE	EACH
P42161	BACKREST BLUE	EACH
P42162	BACKREST RED	EACH
P4215B	BASKET	EACH
VT-33207	BEARING FOR SWIVEL FRONT FORK	EACH
P40006	BEARINGS FOR WHEEL	BAG5
P-CABLE	BRAKE CABLE (PAIR)	PAIR
P40027	BRAKE SHOE	PAIR
P-COVER	CABLE COVER BLACK	EACH
P40013	CRIMP FOR STEEL CABLE (PAIR)	PAIR
P42166	FOAM FOR BACKREST	EACH
P42053	FOLDING MECHANISM	EACH
P421611	FORK FOR FRONT WHEEL	EACH
P40031	FORK FOR REAR WHEEL	EACH
P421615	FRAME ATTACHMENT	EACH
P43003	GRIP - LEFT ANATOMICAL	EACH
P43004	GRIP - RIGHT ANATOMICAL	EACH
B14-C	HAND BRAKE WITH STEEL CABLE	EACH
P42169	HANDLE LEFT SIDE	EACH
P421610	HANDLE RIGHT SIDE	EACH
P421612	HANDLE SPACER	EACH
V420902	HARDWARE FOR FOLDING MECHANISM	EACH
P40015	NUT COVER FOR WHEEL AXLES	EACH
P40028	NUT FOR FOLDING MECH	EACH
P40016	NUT FOR FRONT/REAR WHEEL AXLE	BAG5
P421619	PLUG FOR DETACHABLE BACKREST	EACH
P40047	REFLECTOR FOR BRAKE HANDLE	EACH
P42051	RELEASE PIN FOR DETACHABLE BACKREST	PAIR
P42007	SCREW FOR FOLDING MECH. & SLANTED FRAME PLUG	EACH
P40045	SCREW FOR FOLDING MECHANISM	EACH
P42167	SCREW FOR FRAME ATTACHMENT	EACH
P40004	SCREW FOR HEIGHT ADJ. KNOB	PAIR

	Item Number 4216	
	<b>Mighty Mack</b>	
	SERIAL # BEGINNING WITH "NT"	
ITEM NUMBER	DESCRIPTION	UOM
P40007	SCREW TO ADJUST CABLE TENSION	EACH
P40030	SCREW TO ATTACH BRAKE SHOE TO REAR FORK	EACH
C40006	SCREW TO ATTACH HAND BRAKE	EACH
P40029	SCREW TO ATTACH REAR FORK TO FRAME	EACH
P40001	SCREWS TO ATTACH PLASTIC SEAT	BAG5
P42163	SEAT	EACH
P42164	SEAT PAD	EACH
P421624	SPACERS & WASHER FOR REAR WHEEL	SET
P421623	SPACERS FOR 4216 FRONT WHEEL	PAIR
P40005	STAR HEIGHT ADJUSTMENT KNOB	PAIR
P42181	STEM FOR FRONT FORK	EACH
P421618	U-SHAPE SEAT ATTACHMENT	EACH
P42009	WASHER FOR FOLDING MECHANISM CONTOURED	EACH
P42010	WASHER PLASTIC CONTOURED FOR FOLDING MECHANISM	EACH
P421516	WHEEL BLACK	EACH

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE NOVA PROMISE