



# Zoom 18

Parts Breakdown  
*For models: 4218BL, 4218DB, 4218PK, 4218PL, 4218RD  
Serial # Starting "NV"*



CARE • QUALITY • INNOVATION



THE NOVA PROMISE

*Revised on January 31, 2023*

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).

If you need further assistance, please call our customer service team at **(800) 557-6682**.

**Thank you!**

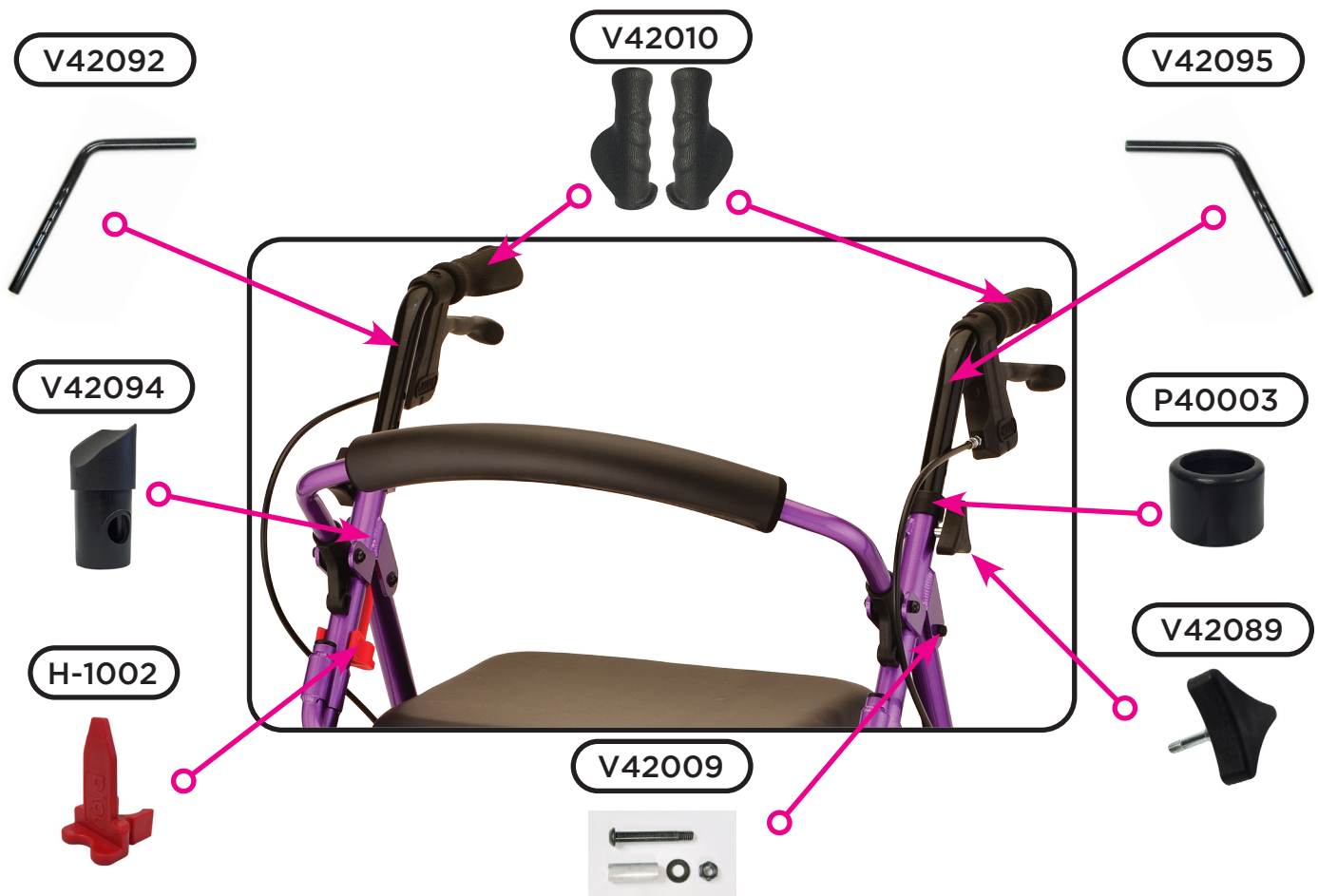


**Figure 1**


## Hand Brake Mechanism



## Upper Assembly Components



## Seat & Backrest

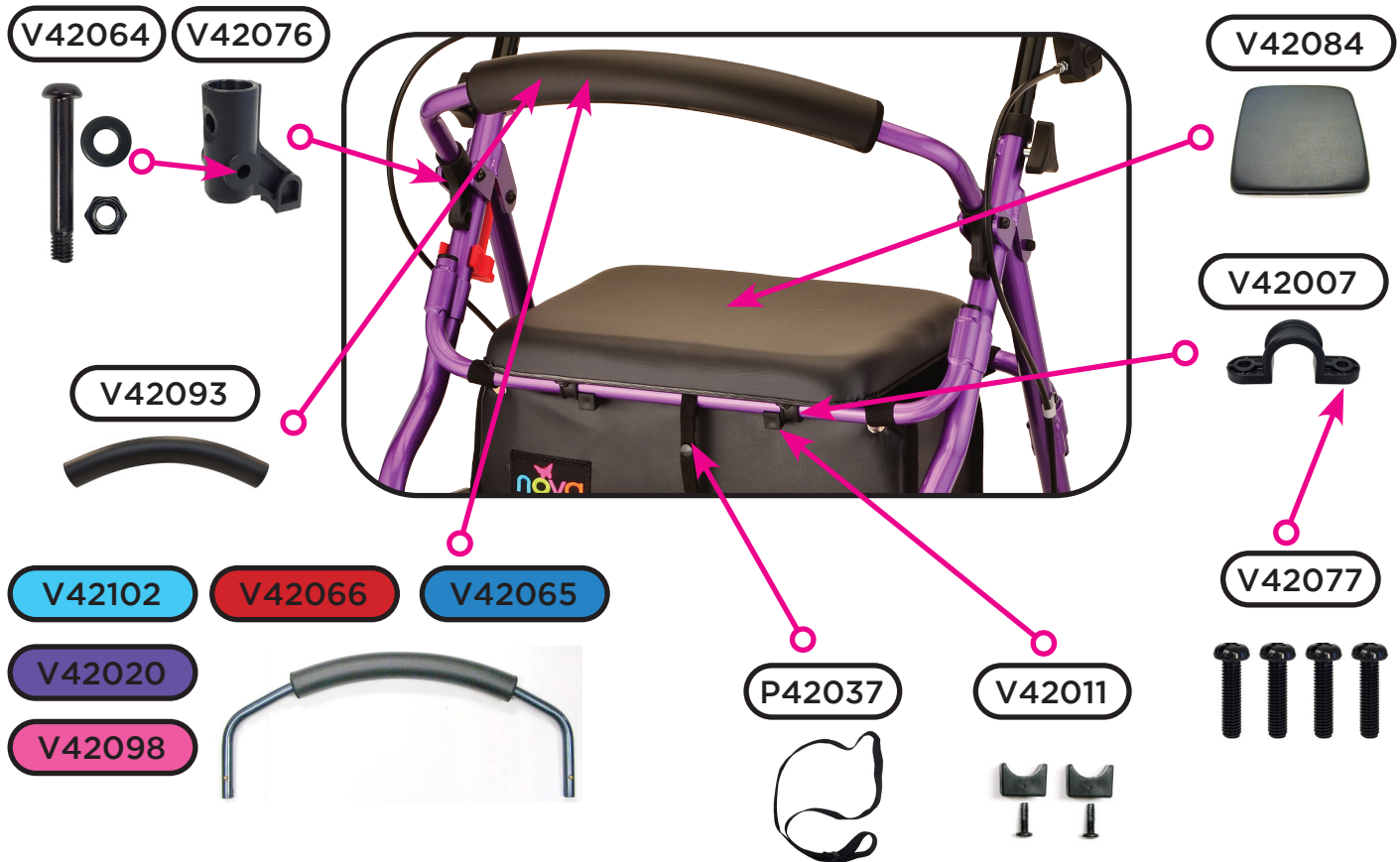


## Folding Mechanism & Accessories





# Wheel & Lower Assembly Components

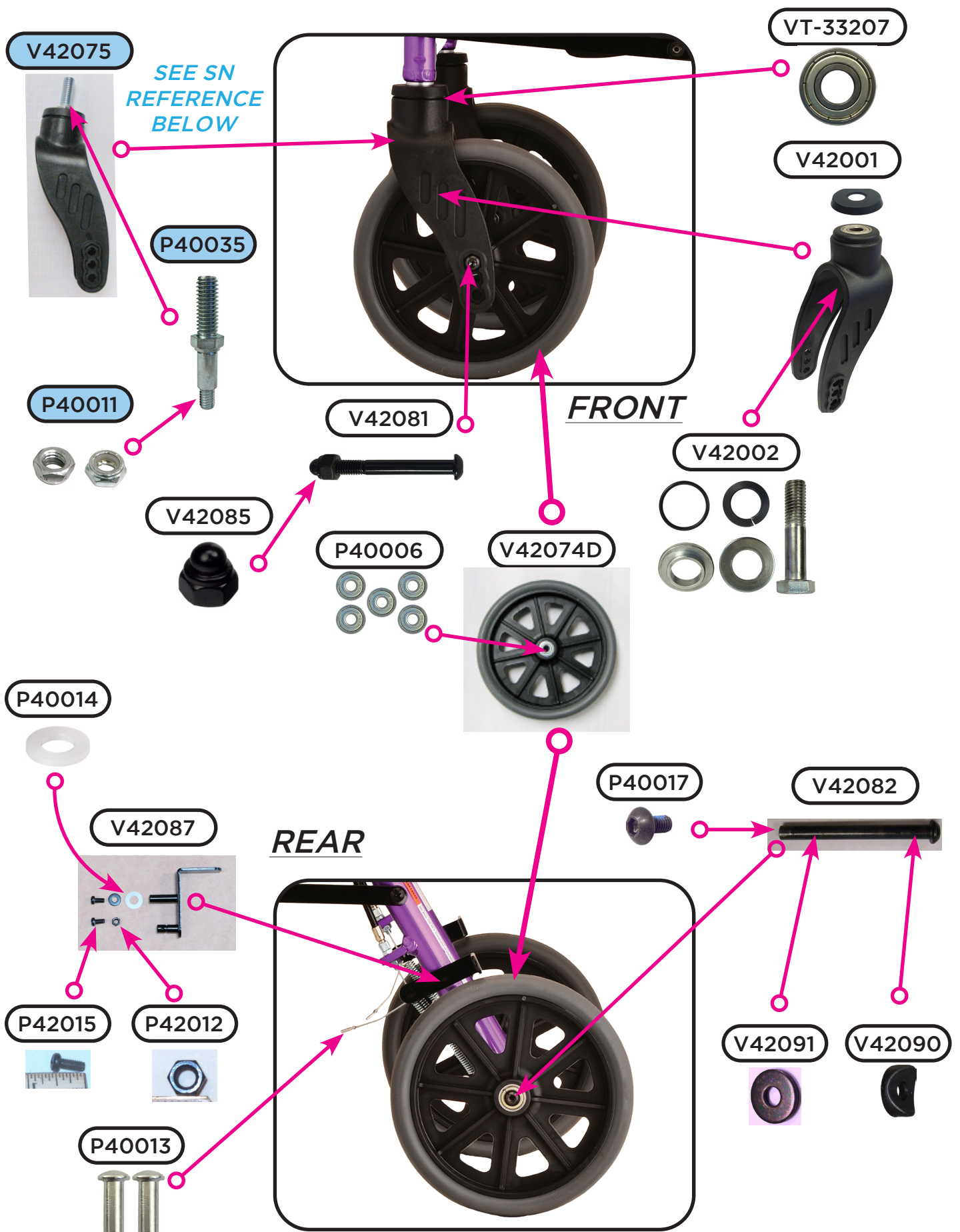




Item Number 4218

## Zoom 18

SERIAL # BEGINNING WITH "NV"


ITEM NUMBER	DESCRIPTION	UOM
V42081	AXLE FOR FRONT WHEEL	EACH
V42082	AXLE FOR REAR WHEEL- FEMALE	EACH
P40017	AXLE REAR WHEEL - MALE	EACH
V42065	BACKREST BLUE FLIP-BACK	EACH
V42102	BACKREST FLIP SKYBLUE	EACH
V42098	BACKREST PINK FLIP-BACK	EACH
V42020	BACKREST PURPLE FLIP-BACK	EACH
V42066	BACKREST RED FLIP-BACK	EACH
VT-33207	BEARING FOR SWIVEL FRONT FORK	EACH
P40006	BEARINGS FOR 5"/6"/8" WHEEL (BAG OF 5)	SET
P42037	BLACK STRAP FOR FOLDING MECHANISM	EACH
P-CABLE	BRAKE CABLE (PAIR)	PAIR
V42087	BRAKE SHOE	EACH
V-COVER	CABLE COVER BLACK	EACH
P40013	CRIMP FOR STEEL CABLE (PAIR)	PAIR
V42093	FOAM FOR FLIP-UP BACKREST	EACH
V42088	FOLDING MECHANISM	EACH
V42001	FORK FRONT	EACH
V42075	FORK FRONT <i>PRE FEB. 2015</i>	EACH
B14-C	HAND BRAKE WITH STEEL CABLE	EACH
V42010	HAND GRIP ANATOMICAL (PAIR)	PAIR
V42095	HANDLE BAR LEFT	EACH
V42092	HANDLE BAR RIGHT	EACH
H-1002	LOCK N LOAD	EACH
P40028	NUT FOR AXLE, FOLDING MECH. AND A-FRAME	EACH
P42012	NUT FOR BRAKE SHOE SCREW	EACH
P40011	NUT FOR FORK STEM <i>PRE FEB. 2015</i>	EACH
V42085	NUT FOR FRONT AXLE	EACH
V42094	PLUG FOR SLANTED FOLDING FRAME	EACH
V4218N	POUCH W/INNER POCKET	EACH
V42076	RECEIVER FOR REMOVABLE BACKREST	EACH
P40047	REFLECTOR FOR BRAKE HANDLE	EACH
V42064	SCREW & NUT FOR BACKREST	EACH
V42077	SCREWS TO ATTACH PADDED SEAT	SET

	Item Number 4218	
	<b>Zoom 18</b>	
	SERIAL # BEGINNING WITH "NV"	
ITEM NUMBER	DESCRIPTION	UOM
P42015	SCREW FOR BRAKE SHOE	EACH
P42007	SCREW FOR FOLDING MECH. & SLANTED FRAME PLUG	EACH
V42009	SCREW SET FOR A-FRAME	SET
P40007	SCREW TO ADJUST CABLE TENSION	EACH
P40035	SCREW TO ATTACH FORK TO FRAME <i>PRE FEB. 2015</i>	EACH
C40006	SCREW TO ATTACH HAND BRAKE	EACH
V42007	SEAT CLAMP	EACH
V42084	SEAT- PADDED	EACH
P40003	SILENCER FOR ADJ. HANDLE	EACH
V42090	SPACER CONTOURED FOR REAR WHEEL	EACH
V42091	SPACER FOR REAR WHEEL AXLE	EACH
P42013	SPRING FOR BRAKE (PAIR)	PAIR
V42002	STEM & WASHERS FOR V42001 FORK	EACH
V42011	STOPPER FOR SEAT (PAIR)	PAIR
V42089	TRIANGLE SHAPED HEIGHT ADJUSTMENT	EACH
P42009	WASHER FOR FOLDING MECHANISM	EACH
P42010	CONTOURED WASHER FOR FOLDING MECHANISM	EACH
V42074D	WHEEL 8"	EACH
P40014	WHITE PLASTIC WASHER FOR BRAKE SCREW	EACH

SERIAL NUMBER (SN) REFERENCE	
ITEM NUMBER	<i>PRE FEB. 2015 MODELS</i>
4218BL	<b>SN BEFORE:</b> NV4218BLDM0001
4218PK	<b>SN BEFORE:</b> NV4218PKDM0021
4218RD	<b>SN BEFORE:</b> NV4218RDDM0071

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE NOVA PROMISE