



# DRAGON

**Fold & Go Rise UP Rollator**

Parts Breakdown  
For models: 4802BL, 4802PL, 4802RD  
Serial # Starting "HT"



CARE • QUALITY • INNOVATION



THE NOVA PROMISE

CONTACT US!  
We're here to help

✉ [info@novajoy.com](mailto:info@novajoy.com)

☎ 1 800 557 6682

[novajoy.com](http://novajoy.com)

Revised on September 1, 2021

## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (see Figure 1).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

**Thank you!**




**Figure 1**



## Upper Assembly Components

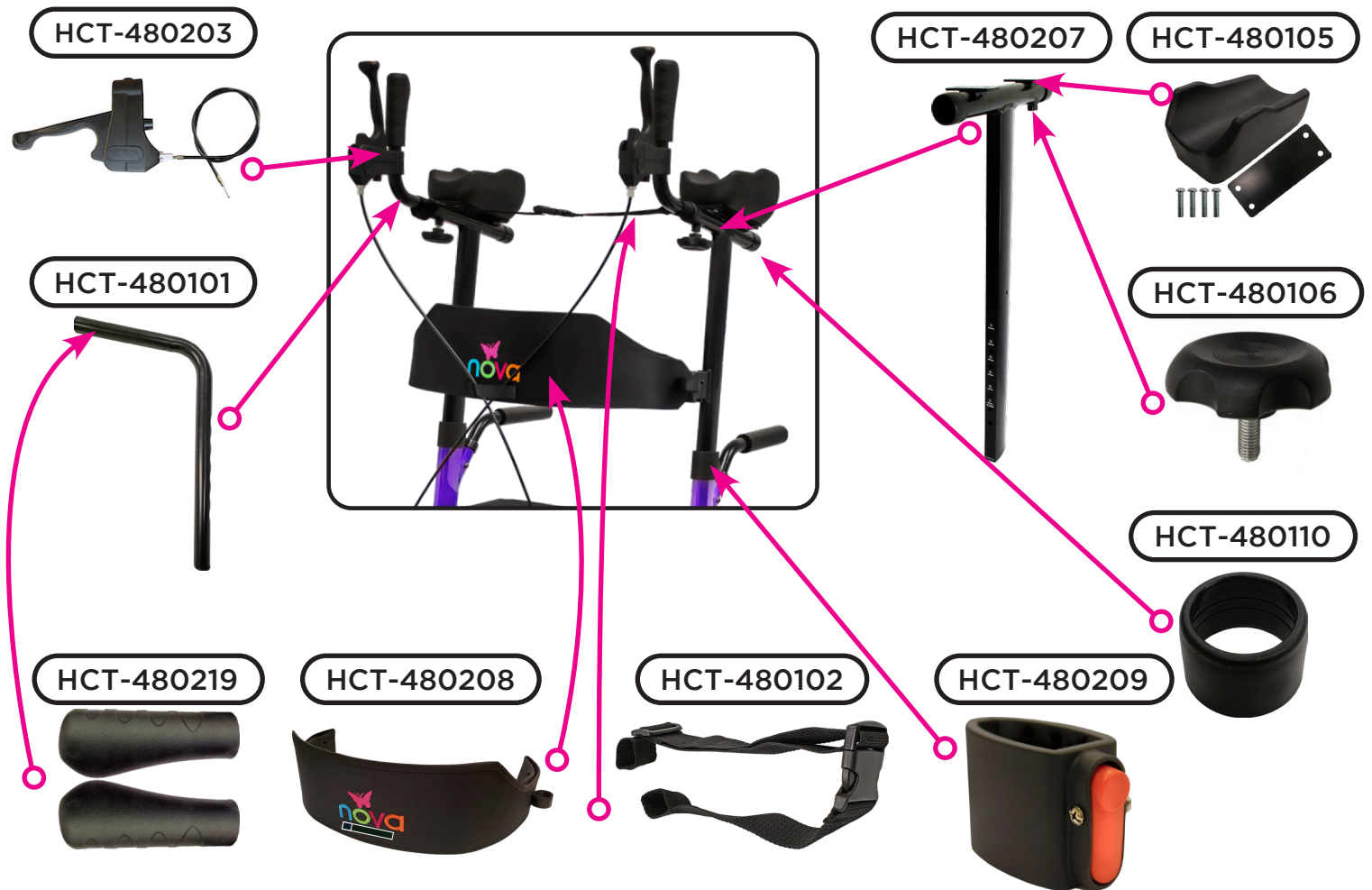

## Seat & Backrest

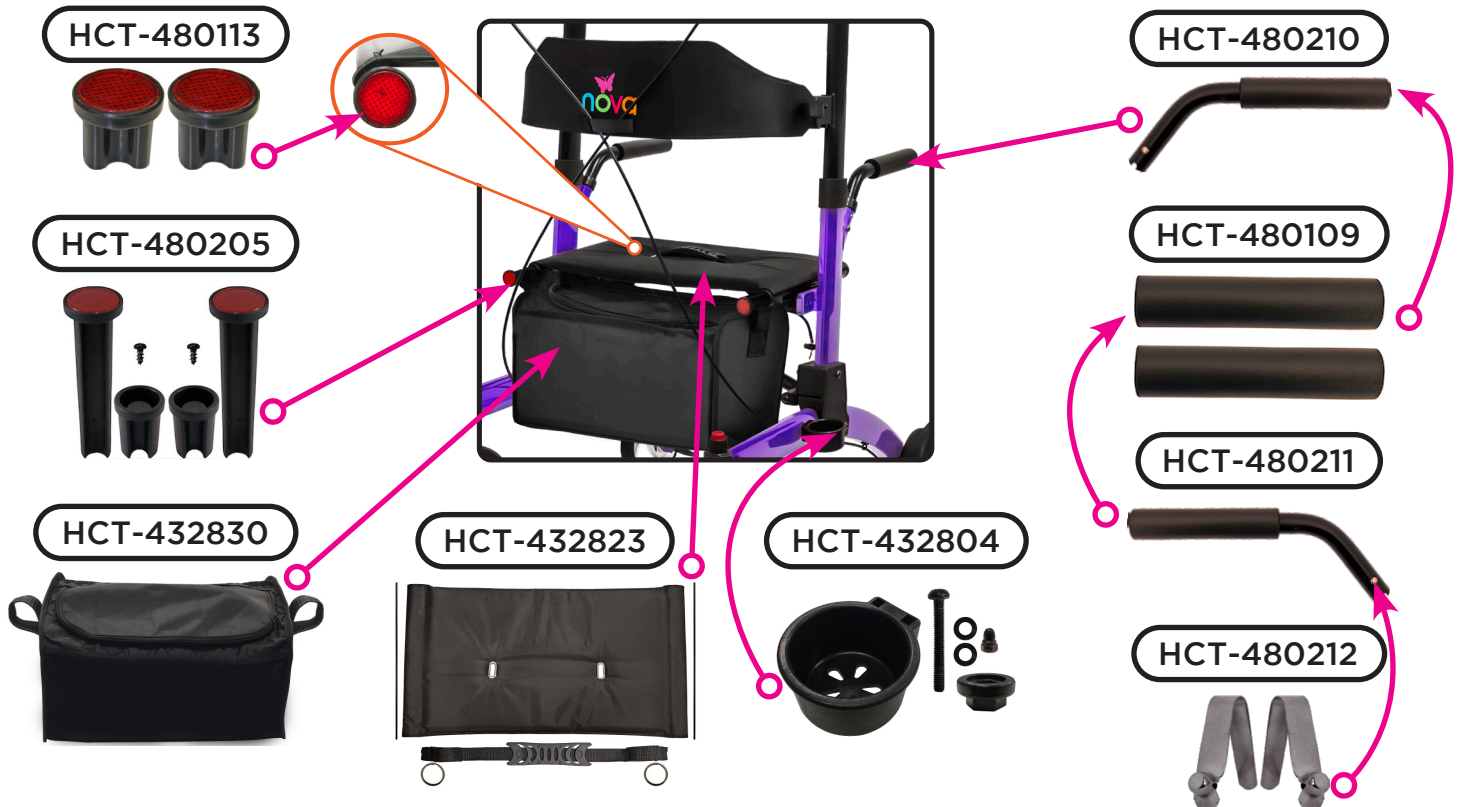


# Folding Mechanism Componets

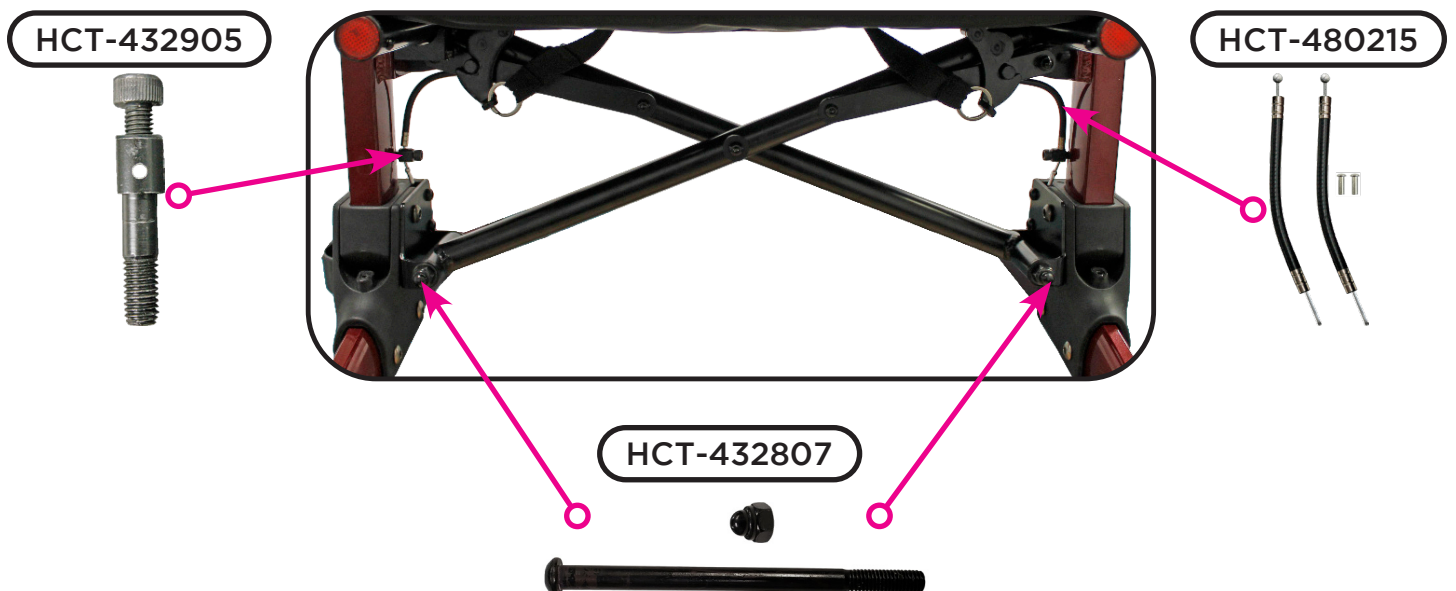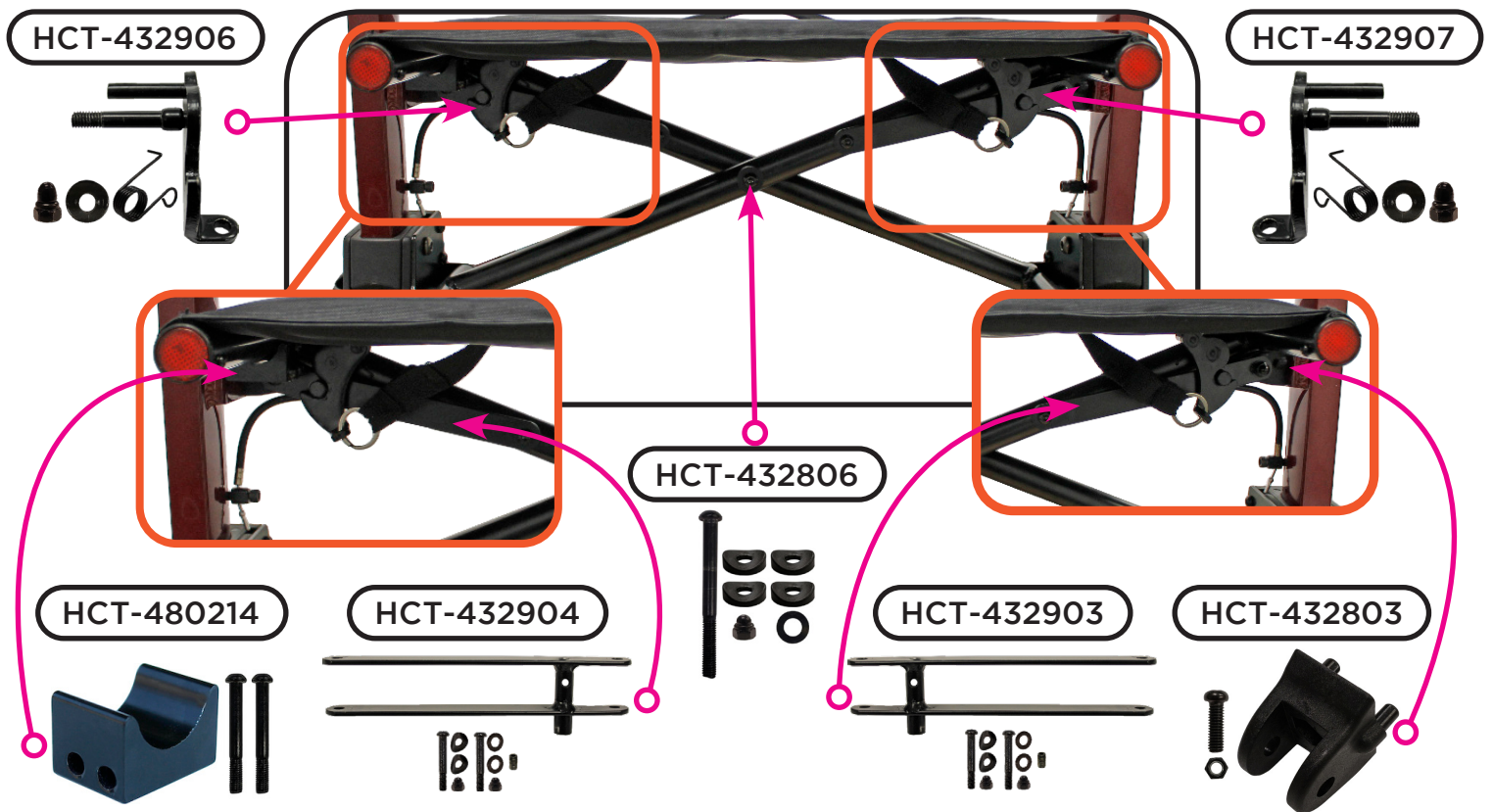





## Fork Assembly Components

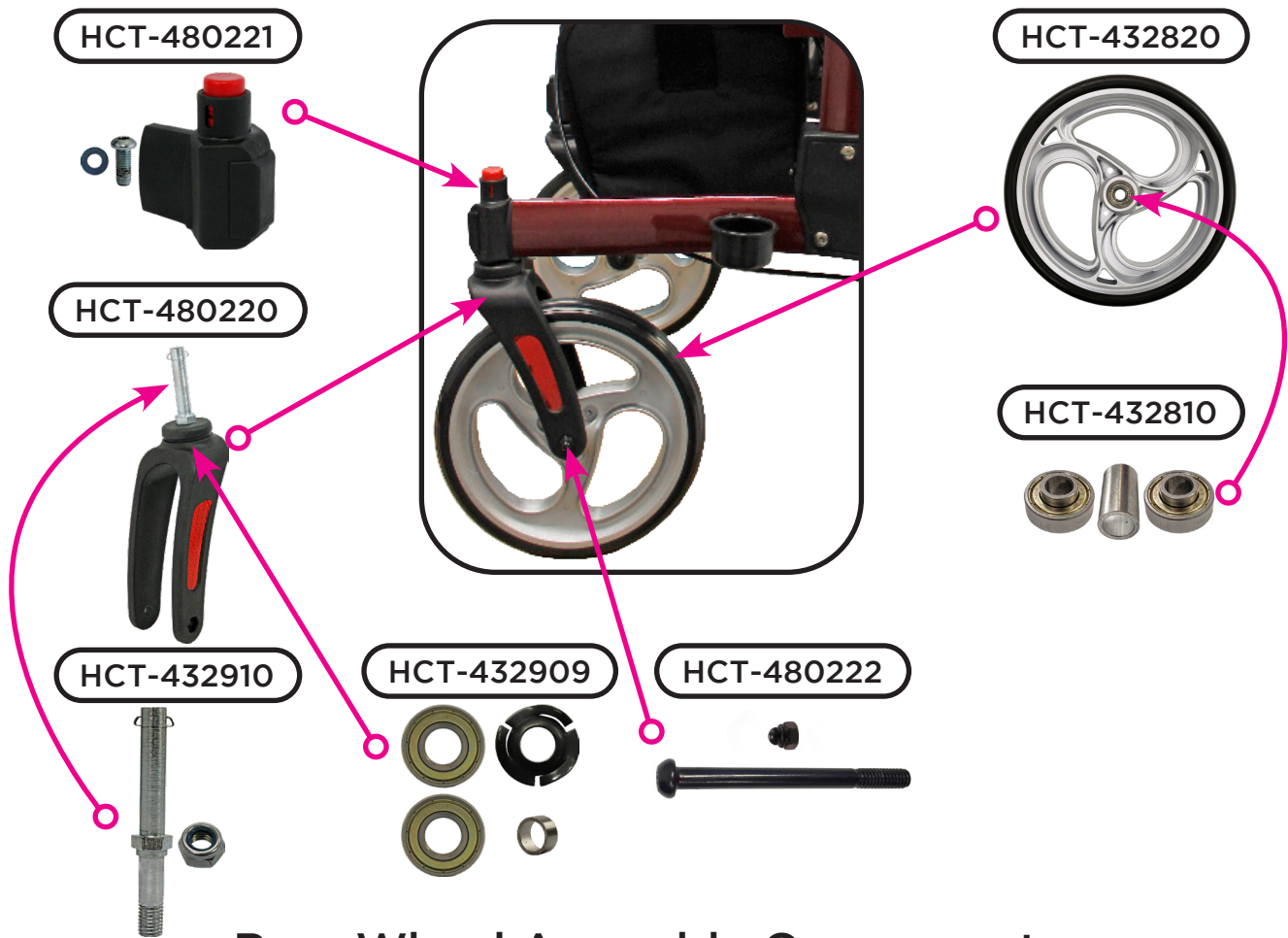

## Rear Wheel Assembly Components

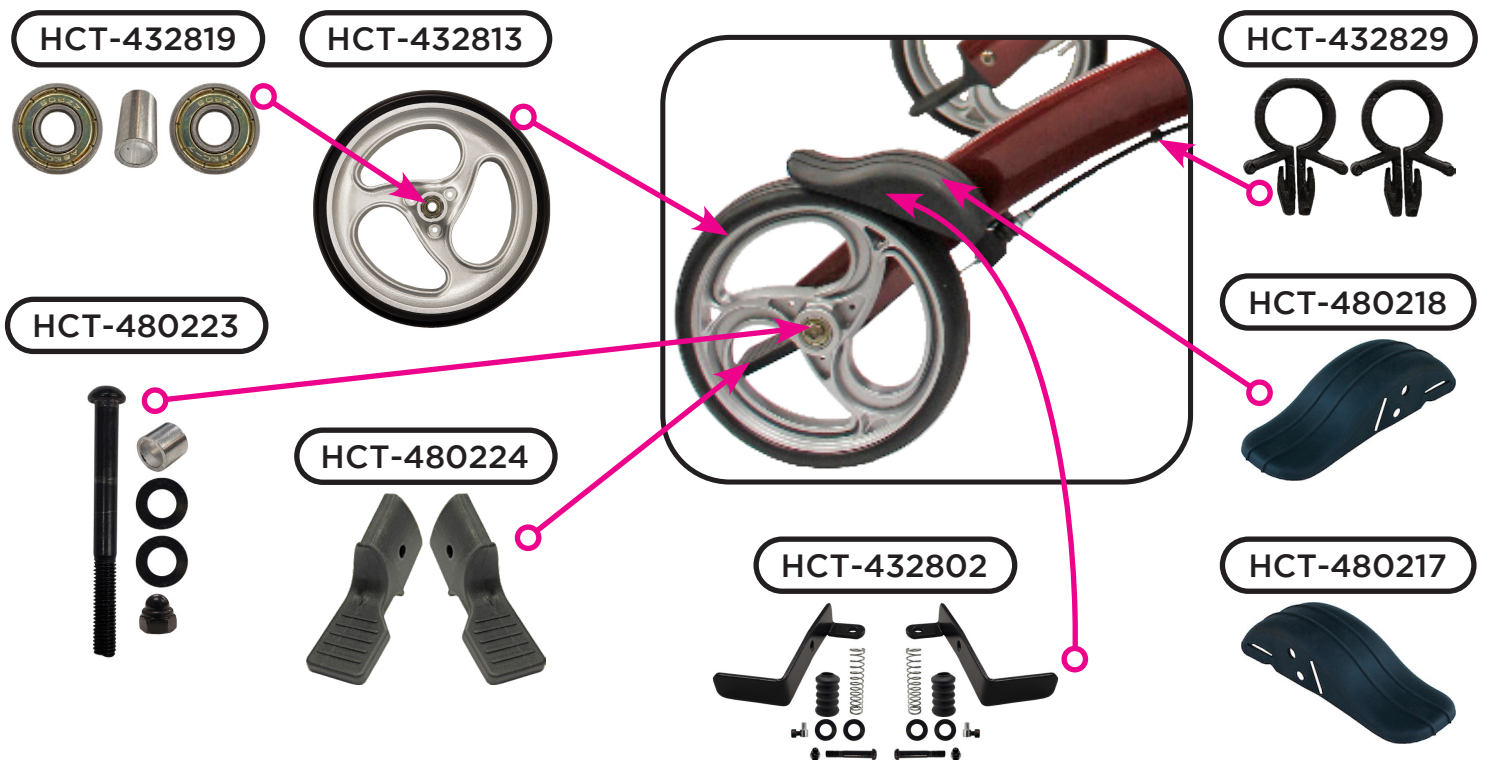

	Item Number 4802	
		
	SERIAL # BEGINNING WITH "HT "	
ITEM NUMBER	DESCRIPTION	UOM
HCT-480105	ARM PAD FOR FOREARM WITH SCREW	EACH
HCT-480222	AXLE FRONT	EACH
HCT-480223	AXLE REAR	EACH
HCT-480208	BACKREST WITH CABLE HOLDER	EACH
HCT-432830	BAG	EACH
HCT-432810	BEARING SET FOR FRONT WHEEL	SET
HCT-432819	BEARING SET FOR REAR WHEEL	SET
HCT-432802	BRAKE SHOE SET	SET
HCT-432804	CANE HOLDER TRAY	EACH
HCT-480217	COVER FOR LEFT BRAKE	EACH
HCT-480218	COVER FOR RIGHT BRAKE	EACH
HCT-432806	CROSSBAR CENTER BOLT	EACH
HCT-432807	CROSSBAR SIDE BOLT	EACH
HCT-432803	CROSSBAR SIDE BRACKET	EACH
HCT-432903	CROSSBAR SUPPORT LEFT	EACH
HCT-432904	CROSSBAR SUPPORT RIGHT	EACH
HCT-480109	FOAM GRIPS FOR SEAT HANDLES	PAIR
HCT-480207	FOREARM HANDLE	EACH
HCT-480220	FORK	EACH
HCT-480221	FORK ATTACHMENT	EACH
HCT-480219	GRIPS FOR HANDLE BAR	PAIR
HCT-480203	HAND BRAKE WITH COVER & CABLE	EACH
HCT-480101	HANDLE BAR	EACH
HCT-480210	HANDLE BAR LEFT FOR SEAT	EACH
HCT-480211	HANDLE BAR RIGHT FOR SEAT	EACH
HCT-480102	HANDLE STRAP	EACH
HCT-432829	HOLDER FOR BRAKE COVER	EACH
HCT-480106	KNOB FOR HANDLE ADJUSTMENT	PAIR
HCT-480212	PUSH BUTTON FOR LOWER HANDLE BAR	EACH
HCT-432906	RELEASE LEVER LEFT	EACH
HCT-432907	RELEASE LEVER RIGHT	EACH
HCT-432905	SCREW WITH BARREL FOR SEAT CABLE & COVER	EACH
HCT-480215	SEAT CABLE & COVER	EACH
HCT-480214	SEAT CRADLE	EACH

	Item Number 4802	
		
	SERIAL # BEGINNING WITH "HT "	
ITEM NUMBER	DESCRIPTION	UOM
HCT-480205	SEAT PLUG FRONT	PAIR
HCT-480113	SEAT PLUG REAR	PAIR
HCT-432823	SEAT WITH HANDLE	EACH
HCT-480110	SILENCER FOR FOREARM HANDLE	PAIR
HCT-480209	SILENCER WITH RELEASE BUTTON	EACH
HCT-432910	STEM WITH NUT FOR FORK	EACH
HCT-432909	SWIVEL BEARING AND SPACERS FOR FORK	SET
HCT-480224	TOE KICK	SET
HCT-432820	WHEEL 10" FRONT	PAIR
HCT-432813	WHEEL 8" REAR	EACH


CARE • QUALITY • INNOVATION



THE NOVA PROMISE

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)