Our mission is to change the world - every day - every way

## 377-R User Guide

User Guide and video available online at [www.novamedicalproducts.com](http://www.novamedicalproducts.com)

**GO!**  
MOBILITY

### Specifications/Dimensiones

Weight Capacity/Peso Max. Usuario	250 lbs.
Weight/Peso	19 lbs.
Seat Width/Anchura de Asiento	18"
Seat Depth/Profundidad del asiento	16"
Seat Distance from Floor/ Distancia del asiento al piso	19"
Arm Height from Seat/Altura total (del brazo al asiento)	9"
Overall Height/Altura total	37.5"
Overall Width (Opened)/Anchura total	23"
Overall Width (Closed)/Anchura Plegada	8.5"

### Features

- Lightweight aluminum compact frame weighs only 19 lbs. (without footrests)
- Large 8" wheels provide increased maneuverability
- Provides quick and easy transfer for shopping, appointments or sporting events
- Quick release fold down back allows the chair to be transported and stored in compact areas
- Quick adjust footrests swing away for safe patient transfer
- Removable wheels allow chair to fit into a convenient travel bag (21.25"l x 22.5"w x 9.5"d)
- Padded armrests
- Limited Lifetime Warranty

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## Safety Warnings

- User weight not to exceed 250 lbs.
- This transport chair has not been approved as a seat surface to be used within a vehicle.
- Do not operate this transport chair on streets or roadways.
- Do not attempt inclines unassisted.
- Engage the wheel locks upon entering or leaving the transport chair.
- Transport chair must be on a level surface when transferring user to or from seat.
- Footrests should be no less than 2 1/2 inches from the floor.
- Do not stand on or put pressure on footrests when entering or exiting the transport chair.
- Do not lean over the top of the chair back or to the side of the chair; this can cause the transport chair to tip over.
- Do not use the wheel locks to slow your descent down an incline.
- When going down an incline, changing directions can cause instability.
- Always test the push handles to make sure that both push pins are securely in place before sitting in the chair.
- Only operate the chair with the push handles in the upright locked position.
- Do not scoot on or operate the chair without an assistant maintaining a secure grip from behind the chair.
- Always secure the user to the chair using the seat belt.
- DO NOT push down on the rear handles to attempt to raise the front wheels up when user is in the chair. This could result in the breaking of the folding release back and the user falling backwards.

### WARNING

Failure to follow any or all safety instructions may result in serious injury or death.

- DO NOT ADJUST, MAINTAIN, OR OPERATE THIS TRANSPORT CHAIR WITHOUT READING, UNDERSTANDING, AND FOLLOWING, THE PROPER INSTRUCTIONS INCLUDED WITH THIS PRODUCT

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Frame  
--  
marco de la silla

## Assembly Instructions

- Step 1:** Carefully remove all the components from the carton and examine all parts for damage from shipping. If any parts are missing or damaged please call Nova Customer Service at **1-800-557-6682** and **press 1**. DO NOT use the transport chair until all parts are received.
- Step 2:** Place the frame on a non scratch surface and flip the folding push handles up until the push pins engage into the frame.
- Step 3:** Open the frame by pulling outward on the padded armrests. **(see image 1)**
- Step 4:** Lean the chair backwards so that it rests on the ends of the folding push handles.
- Step 5:** Insert the front castor stem into the frame by depressing the push pin in and sliding the stem in until the push pin engages into the frame hole. **(see image 2)** Repeat for second assembly
- Step 6:** Insert the castor assembly screw into the frame and through the castor stem. Tighten the screw until it is fully secure and flush against the frame. **(see image 3)** Repeat for the second assembly.
- Step 7:** Turn the frame over so that it rests on the front of the frame. **(see image 4)**
- Step 8:** Select the rear right castor assembly (wheel and brake go on the outside of the frame) and insert it into the frame by depressing the push pin in and sliding the stem in until the push pin engages into the frame. **(see image 5)** Repeat for the second assembly.
- Step 9:** Insert the castor attachment screw into the frame and through the stem and tighten until the plastic knob is secure and flush against the frame. **(see image 6)** Repeat for the second assembly.
- Step 10:** Attach the footrests by inserting the hinge pins into the holes on the hinge plate. **(image 11)** Swing the footrest forward until the release lever locks into place. **(image 12)** Repeat for opposite side.

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# User Instructions

## To Sit Down

- Step 1:** Engage the rear wheel locks by pushing down on the brake handle until the brake shoe presses firmly on the wheel and prevents the chair from rolling.
- Step 2:** Flip the footrests into an upright position so that the user can stand between the footrests with the back of her/his legs touching the seat upholstery.
- Step 3:** From behind the chair, assist the user by slowly lowering them into the chair.

## To Rise from Seated Position

- Step 1:** Engage the rear wheel locks located above the wheel by pushing down on the brake handle until the brake shoe presses firmly on the wheel and prevents the chair from rolling.
- Step 2:** Flip both footrests into an upright position.
- Step 3:** Place both feet firmly on the ground and rise slowly from the chair with the assistance of a caregiver.

## To Transfer from Seated Position

- Step 1:** Place the chair alongside the desired seating device.
- Step 2:** Engage the rear wheel locks located above the wheel by pushing down on the brake handle until the brake shoe presses firmly on the wheel and prevents the chair from rolling.
- Step 3:** Flip the both footrests into an upright position.
- Step 4:** Carefully transfer from the chair to desired seating area with the assistance of a caregiver.

## To Fold

Grab the nylon straps located on the side of the seat and pull upward. These straps can also be used to carry the chair. (see image 7)

## To Unfold

Push the seat down and outward on the seat just inside of the seat support tubes (see images 8 & 9).

## To Release the Folding Push Handles

Push down on the release lever to disengage the push pin from the frame and gently lower the handle down. (see image 10)

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image 10

### General Care

Wipe all metal parts at least once a week using a soft cloth. Dry chair immediately if it is exposed to moisture. Abrasive cleaners or materials will scratch the surface of the chair.

### Tires

Minimal maintenance is required for rubber tires. Wipe with a damp cloth occasionally. If tires become cracked or worn, replace immediately.

### Front Castors

Inspect castors periodically and lubricate them when necessary. To lubricate, remove the fork and grease the castor bearings. Remove the wheel from the fork and grease the wheel bearings. The wheel and fork can be reassembled by tightening the nuts until there is no play remaining, but the wheel rolls freely. The fork will revolve freely when adjusted correctly. The castor will flutter if the nut is too loose or be difficult to steer if the nut is too tight.

### Castor Attachment Screws

Periodically check to make sure the screws are tight and securely holding the castor assembly to the frame.

### Wheels

Inspect monthly for correct tightness. If the wheel seems to be loose or wobbly, release the lock nut and tighten the axle bolt. Lean the chair to one side and spin adjusted wheel to check for correct adjustments.

### Wheel Locks

Wheel locks should be checked for correct locking action before use. Pushing the handle completely forward activates the wheel locks. If the wheel locks are working correctly they will be embedded into the tire at least 1/8". To adjust wheel locks, loosen nut on carriage bolt. Slide clamp toward rear wheel until lock shoe is embedded at least 1/8" into rear tire when lock handle is engaged. Tighten nut to secure into position.

- Do not attempt to repair the transport chair yourself.
- Call the store where you purchased the transport chair. Most dealers do repairs and will order parts for you.
- If your dealer is unable to help you, please call Nova at **1-800-557-6682**.

## Warranty Information

Nova stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product. Nova extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. Nova warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

<b>Metal Frame:</b>	<b>Limited Lifetime Warranty</b>
<b>Non Metal/Metal Parts:</b>	<b>One Year Warranty</b> (Wheels, Footrests, Armrests, Upholstery, hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your Nova product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

**Customer Help Line:**  
**1-800-557-6682**  
**www.NovaMedicalProducts.com**  
**P.O. Box 3039, Gardena, CA 90247**

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