

## HOW TO ASSEMBLE YOUR



### Toilet Seat Riser - Hinged



**No tools required  
Quick Assembly**

**Follow these  
SIMPLE STEPS**

Item #'s 8345-R /8346-R  
Capacity not to exceed 300 lbs.



**1** Remove all parts from box:  
One seat, 2 mounting screws, 2 washers and 2 wing nuts  
Clean and sanitize toilet seat.

**2** Remove the existing toilet seat and lid from the toilet frame by unscrewing the seat screws located at the rear of the seat.

**3** Place the toilet seat elevator on the toilet frame. The mounting holes on the rear of the elevator should line up with the mounting holes on the rear of the toilet frame.

**4** Replace the toilet seat and lid so the mounting holes of the toilet seat, the elevator and the frame line up. Place washer onto screw. Insert the mounting bolts with washer through top of the seat, seat elevator, and toilet frame.



**5** Attach the provided wing nuts to each mounting screw. Do not tighten completely. Be sure the toilet seat is straight and sits flush on the seat and then fully tighten the wing nuts.

**6** To raise elevator seat lift using side openings

**IMPORTANT WARNING**  
**READ THESE DIRECTIONS CAREFULLY BEFORE USE.**  
**FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

- Be sure seat is in place and secure before each use.
- Do not stand on seat.

Questions?  
Please contact us at  
1-800-557-6682  
info@novamedicalproducts.com

Thank you for choosing NOVA!

Connect with us!  
novamedicalproducts.com  
facebook: novamedicalprod  
twitter: novamedicalprod  
twitter: novawalkertalk  
pinterest: novamedicalprod

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

Regular cleaning is recommended. Use a non abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. please regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.