## HOW TO ASSEMBLE YOUR

### DROP ARM TRANSPORT CHAIR COMMODOE

Item # 8805

Capacity not to exceed **250 lb**

**NO tools required**  
**Quick assembly**

**Follow these**  
**SIMPLE STEPS**

#### WHAT'S INSIDE

*Be sure all parts are included before assembling.*

- |                             |                                       |
|-----------------------------|---------------------------------------|
| a. (1) Chair frame          | c. (1) Backrest with tightening knobs |
| b. (1) Chair seat w/ bucket | d. (1) Seat pad                       |
|                             | e. (1) Foot rests and heel strap      |

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#### **WARNING**

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.**

- **DO NOT** use commode inside shower. This product may result in damage if exposed to excessive moisture.
- **DO NOT** use commode without first reading and understanding these instructions and warnings.
- **DO NOT** use if any parts are cracked, warped, or damaged.
- Ensure all parts are correctly assembled and all screws and nuts are secure and fully tightened before using.
- Ensure all brakes are properly engaged before attempting to sit down on or stand up from unit.

#### **CARE AND MAINTENANCE**

Check regularly to make sure the knobs are fully tightened before using. Use a mild detergent and warm water to clean and thoroughly dry with soft cloth. Regularly inspect your item and its parts for any corrosion, breakage, compression, or wear. **DO NOT** use if product is damaged.

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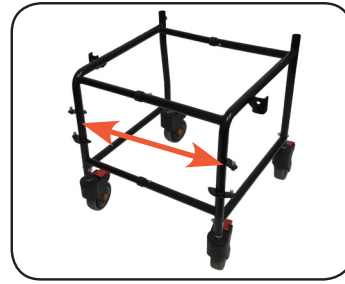
# 1

Set chair frame on flat surface. Spread frame apart by pulling on the longer sides of the frame. Pull apart until the 3 gold push buttons are fully engaged (located in the 3 cross bars). Test for stability and make sure all push pins are fully engaged.

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# 2

Place the commode seat with bucket onto the chair frame. Push down on commode seat until the seat snaps onto frame and stabilizes. (Black seat pad may be placed on top of commode seat).

**To remove commode bucket, slide out bucket towards the rear of the seat.**

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# 3

Insert the backrest into the vertical tubing of chair frame.

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# 4

**Pull up on the arms** of backrest until the push pins on arms into designated holes (pull on black knob to release armrest into downward position).

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# 5

Insert the 2 black knob screws into the screw holes where the backrest frame meets the seat frame. Tighten and test for stability.

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### **Foot Rests and Heel Strap**

**To put on footrests:** Place both top and bottom pin holes of footrest onto the designated pins towards the side (90 degrees from the front of the chair), swing the footrest towards the front until it clicks into place.

**To take off footrests:** Push the small black lever and allow the footrests to swing out 90 degrees, lift directly upwards to remove.

**Heel Strap:** To install, wrap the ends of the strap around the leg of each installed foot rest. Secure each side of strap by connecting the Velcro pieces.

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### **Brakes**

**To lock brakes:** push down the silver metal tab on both sides of back wheels (make sure brakes are locked before sitting or attempting to stand up).

**To unlock brakes:** pop the silver tab up by pressing down on the smaller raised tab.

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---

## **WARRANTY INFORMATION**

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

**Metal Frame**

**Non-Metal/Metal parts**

**Limited Lifetime Warranty**

**One Year Limited Warranty**

**(rubber tips, plastic or metal hardware)**

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**IMPORTANT WARNING**

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

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