

# HOW TO ASSEMBLE YOUR



## HYGIENIC BATH SEAT WITH BACK AND ARMS


Item # 9037, 9037-R

Capacity not to exceed 300 lb



NO tools required  
Quick assembly


Easy to follow  
step-by-step  
instructions

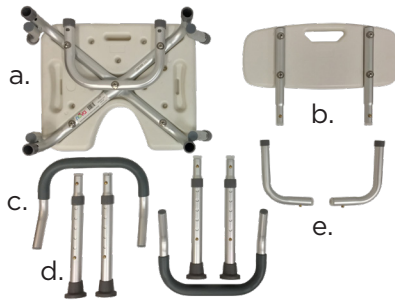

### WHAT'S IN THE BOX


- a. (1) Seat
- b. (1) Backrest
- c. (2) Arms
- d. (4) Adjustable legs
- e. (2) L-shaped back tubes

1

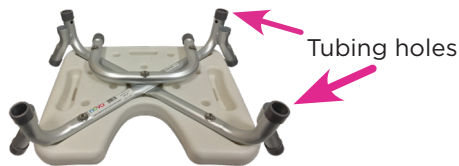


Place the seat face down on a level surface. Insert the four legs into the tubing holes. Depress the push button and push the tube in until the button “clicks” and locks into position.

2

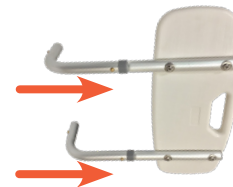

Insert the L-shaped back tubes into the backrest tubing holes until the push button “clicks” into locked position.

3

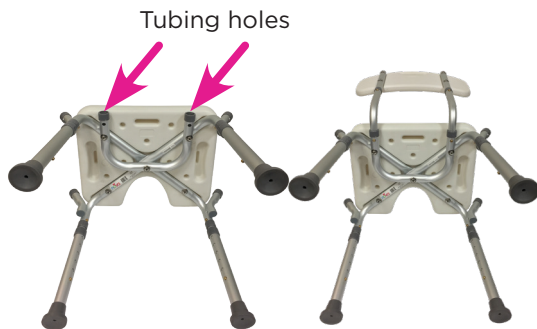

Turn the chair on its side so that the tubing holes are facing upward. Insert backrest into tubing holes until the push buttons “click” and lock into position.

4



Place the chair upright so that it is resting evenly on all four legs and insert the arms into the frame. Depress both push buttons and push down until the push buttons “click” and lock into position. Repeat this step for the opposite arm.

5



Use the push buttons and numbered holes to adjust the seat to the desired seat height. Ensure the Bath Seat is level, stable, and all push buttons are fully engaged before using.

**NOTE:** Arms should curve inwards towards each other.

## **WARNING**

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.**

- Chair legs must be adjusted to same height (number) so the bath seat sits level
- Make sure legs and backrest are secure (push button locked completely into hole) before using
- Check before each use
- Use on a level surface
- **DO NOT stand on Bath Seat**

## **HOW TO USE**

Use the Bath Seat when support is required in bath or shower. Arms may be used to lower or raise oneself up from and into the chair. Use the arms for stability and balance when needed.

## **CARE AND MAINTENANCE**

Check regularly to make sure the all brass buttons are securely locked in holes and that the bath seat is stable before sitting. Use a mild detergent and warm water to clean bath seat.

## **WARRANTY INFORMATION**

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	<b>Limited Lifetime Warranty</b>
Non-Metal/Metal parts	<b>One Year Limited Warranty</b> (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

### **IMPORTANT WARNING**

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**CONTACT US!**  
We're here to help


 [info@novajoy.com](mailto:info@novajoy.com)


 1 800 557 6682


[novajoy.com](http://novajoy.com)


*Connect with us!*

[novajoy.com](http://novajoy.com)

 [novamedicalprod](#)

 [novamedicalprod](#)

 [novawalkerstalk](#)

 [NOVA Medical Products](#)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE