

HOW TO ASSEMBLE YOUR



Bath & Shower Seat with Back

Item # 9100-R
Capacity not to exceed 300 lb



Easy to follow step-by-step instructions



Remove all parts from box. Ensure all parts are included before assembling.

- A. (1) Seat
- B. (1) Backrest
- C. (1) Leg Assembly 1
- D. (1) Leg Assembly 2
- E. (1) Red Safety Handle
- F. (2) White Plastic Screws
- G. (1) Screw Set (4 Short Screws (42mm), 1 Long Screw (50mm), 5 Contoured Washers, 1 Ratchet Washer)


TOOL NEEDED:
PHILLIPS SCREWDRIVER

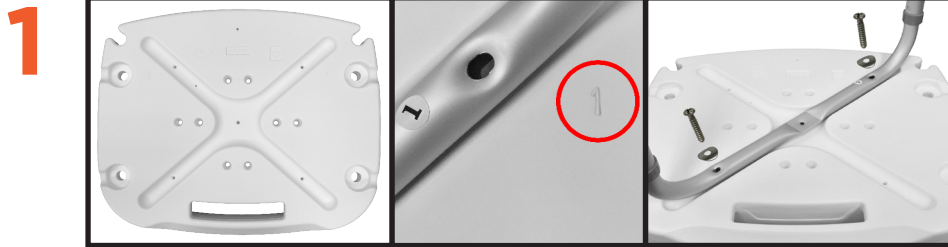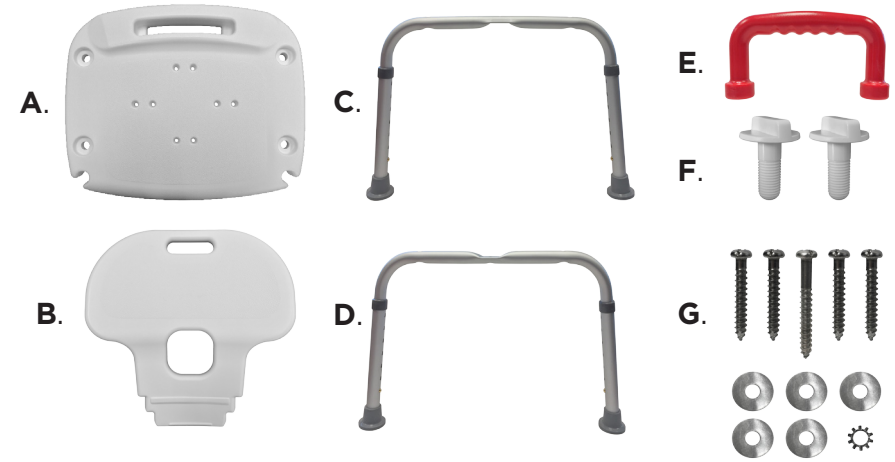

Place the "Seat" upside down on a level surface. Locate the "1" on the bottom of the "Seat". Place "Leg Assembly 1", with the "1" sticker, across the "Seat" so that the "1" on the "Leg Assembly" is across the "1" on the "Seat". Line up the holes on "Leg Assembly" with the holes of the "Seat". Place a "Contoured Washer" over the 2 end holes of "Leg Assembly 1" and screw in 2 "Short Screws" through assembly and "Seat".




Place "Leg Assembly 2" across "Leg Assembly 1" so that they are perpendicular. Ensure the "2" sticker on "Leg Assembly 2" is across the "2" on the "Seat". Place a "Contoured Washer" over the 2 end holes of "Leg Assembly 2" and screw in 2 "Short Screws" through "Leg Assembly 2" and "Seat". Place a "Contoured Washer" over the center hole of "Leg Assembly 2" followed by the "Ratchet Washer" and screw in the "Long Screw". Ensure all 5 screws are tightened.

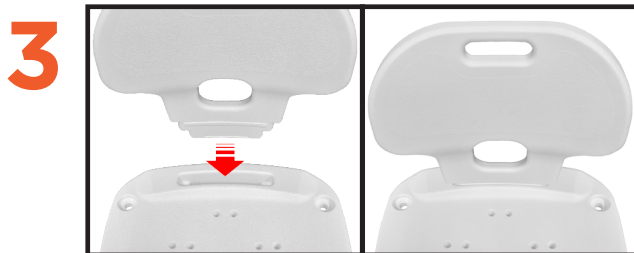

Insert the "Backrest" into the slot located at the back of the "Seat". Ensure that the "Backrest" is completely pushed down and securely locked into position.



Fit the "Red Safety Handle" in either side of the "Seat" into the slot holes. Under the "Seat", screw in and fully tighten both "White Plastic Screws" into the "Red Safety Handle".

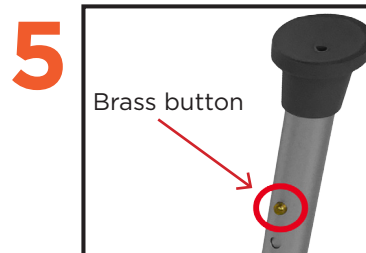

Depress the push buttons on the legs and use the numbered holes to adjust the seat to a desired height. Ensure that each leg is set to the same height, stable, and all push buttons click and lock into position.



HOW TO USE

Place the bath bench in the shower or bath tub. Ensure that the bench is stable, not wobbly. Use the "Red Safety Handle" to assist in sitting down or getting up from the bench. The two round slots in the front corners of the seat can be used to place a hand held shower set if needed. ***Hand Held Shower Set not included***

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame - Limited Lifetime Warranty
Non-Metal/Metal parts - One Year Limited Warranty
(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- All legs must be adjusted to same height (number) so the bath seat sits level
- Make sure legs and backrest are secure (brass button locked completely through hole) before using
- Check before each use
- Use on a level surface
- Do not stand on bath seat

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Connect with us!
novamedicalproducts.com
facebook: [novamedicalprod](#)
twitter: [novamedicalprod](#)
twitter: [novawalkertalk](#)
YouTube: [NOVA Medical Products](#)