


Holder for Hand Held Shower

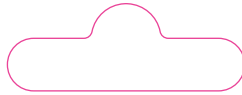

Assembly Instructions

1. Attach to a straight section of tubing on either front leg.
2. Place on leg so cut out rim is on top.
3. Point holder opening at a 45 degree angle away from the seat.

To attach to 1" diameter tube (most bath bench legs)

1. Remove the screws and black spacer from the rear portion of the holder.
2. Place the rear portion of the holder at your desired location.
3. Match the front portion of the holder with the rear portion and insert screws.
4. Tighten with a small Phillips head screwdriver until the holder is secure.



To attach to a 3/4" diameter tube (most bath bench arms)

1. Remove screws from the holder.
2. Place the rear of the holder at your desired location and spread the black spacer edges out so that it wraps around the tube.
3. Match the front portion of the holder with the rear portion and insert screws.
4. Tighten with a small Phillips head screwdriver until the holder is secure.

HOLDER FOR
HAND HELD SHOWER

Item Code 9306-R




UPC CODE



6 52308 13982 3

CONTACT US!
We're here to help

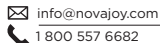

CARE • QUALITY • INNOVATION



nova

NOVA Medical Products
1470 Beachey Place
Carson, CA 90746 USA
novajoy.com
email: info@novajoy.com

MADE IN CHINA

Client:	Nova
Job:	Holder for Hand Held Shower
Size:	4" x 7"
Bleed:	.125"
Color:	
Side A:	4-color process +Pantone 376 (green)
Side B:	Black
Contact:	
	Shelly Freeman 310-408-5934 s.freeman@ucla.edu
Last Edited:	11-18-16