

HOW TO USE YOUR



RUBBER BATH MAT

Item # 9350-R, 9351-R,
9351W-R



NO tools required
Quick installation

Easy to follow
step-by-step
instructions

1 Adding your bath mat to your shower or tub

Rinse mat and clean tub or shower. Wet the base of the tub or shower. Place mat in position with suction cups facing down. Firmly press the mat down to grip the surface and hold in place. Carefully apply pressure to the remaining surface of bath mat until it is securely attached.

2 Care and use

Wash the surface of the mat, and clean tub or shower regularly to prevent build-up of residue that may cause slipping and or injury to the user. Periodically, clean both sides of bath mat using non-abrasive cleaner or detergent. **DO NOT** use any solvents or bleach.

WARNING

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper installation or misuse of this product.**

- **DO NOT** use on uneven or textured surfaces

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Non-Metal parts **One Year Limited Warranty**
(rubber base)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help


 info@novajoy.com


 1 800 557 6682

novajoy.com

Connect with us!

novajoy.com

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkertalk](https://www.youtube.com/novawalkertalk)

 NOVA Medical Products

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

**Revised March 11, 2019*