

# HOW TO ASSEMBLE YOUR



## FOLDABLE SHOWER CHAIR

### WITH BACK

Item # 9400

Capacity not to exceed 250 lb



### WHAT'S INCLUDED

- a. (1) Shower Chair
- b. (1) Backrest



a.




b.

**NO tools required**  
**Quick assembly**


**Follow these**  
**SIMPLE STEPS**

- 1** Unfold the side frames forward so that the front legs are slightly wider than the rear legs. (see **Figure 1**).

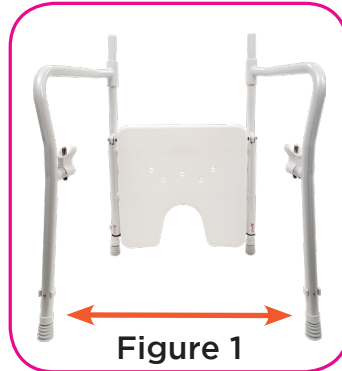


**Figure 1**

- 2** Lift seat up and swing red safety clasps outward on both sides of the chair (see **Figure 2**).

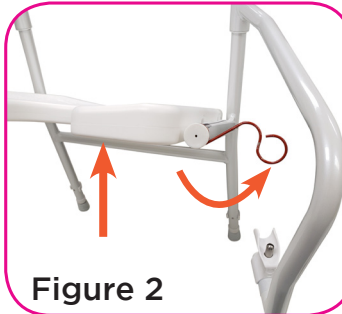


**Figure 2**

- 3** Push legs inward so that the seat support tubes rest on the plastic cradles. Push downward to engage metal pins into the seat support brackets (see **Figure 3**). Pull up gently to make sure pins are in place.

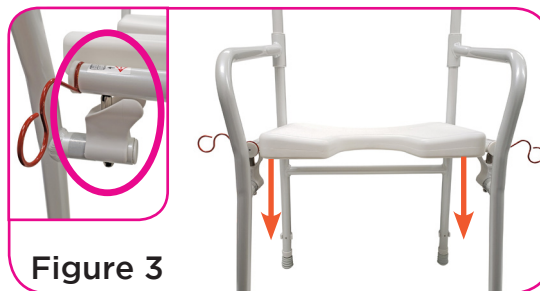


**Figure 3**

- 4** Swing the red safety clasps downward and engage them to the seat support frame to secure seat (see **Figure 4**).

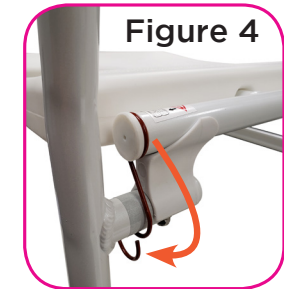


**Figure 4**

- 5** Attach backrest onto frame by guiding the tubing through the holes on each side of the plastic backrest (see **Figure 5**).

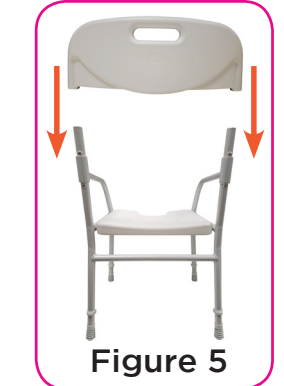

**Figure 5**

- 6** Pull out the leg height adjustment clip. Slide the adjustable leg to the desired height, align the hole of the leg with the hole of the frame, and fully push the center pin of the height adjustment clip through the aligned holes (see **Figure 6**).

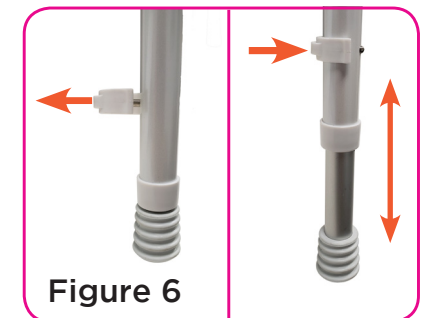

**Figure 6**

## **WARNING**

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. **NOVA Medical Products shall have no liability for any injury resulting from improper assembly or misuse of this product.**

- Ensure ALL leg height adjustment clips are engaged before using.
- All legs MUST be adjusted to the same height to ensure level seating and safety.
- Use EXTREME caution on wet tub or floor surfaces.
- DO NOT stand on shower chair.
- DO NOT use if shower chair is damaged. Check frequently for cracks, damage, or looseness.
- This product is rated for persons weighing up to **250 lb** when properly assembled.

## **HOW TO USE**

Place legs into bathtub. Ensure the shower chair is stable and not wobbly. Adjust leg heights if necessary. Slowly seat yourself onto edge of shower chair, using handles to stabilize as needed.

## **CARE AND MAINTENANCE**

Check regularly to make sure all legs on the shower chair are securely locked. Use a mild detergent and warm water to clean. Dry with a soft cloth to avoid moisture build-up.

## **WARRANTY INFORMATION**

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	<b>Limited Lifetime Warranty</b>
Non-Metal/Metal parts	<b>One Year Limited Warranty</b> (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

### **IMPORTANT WARNING**

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**CONTACT US!**  
We're here to help


 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

*Connect with us!*

[novajoy.com](http://novajoy.com)

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkerstalk](https://twitter.com/novawalkerstalk)

 [NOVA Medical Products](https://www.youtube.com/NOVAMedicalProducts)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE