

## HOW TO USE YOUR



### Foam Cushion



**GREAT FOR TRAVEL**

**ADDS COMFORT  
WHILE SEATED**

Capacity not to exceed 250 lbs.

Item Numbers: 2647C-R, 2651-R, 2652-R, 2653-R,  
2654C-3, 2655C, 2664-3, 2670-R, 2671-R, 2672, 2673 & 2674.

- 1 The flat surface of cushion should be facing down and the rounded surface facing up. For Coccyx cushions (2647C-R, 2654C-3 & 2655C models only), place the U shaped cutout in the rear of the seat so that the cutout is directly under the tailbone area with NOVA logo facing up and on the side.



## CARE INSTRUCTIONS FOR YOUR Foam Cushion

- 1 The cover can be easily removed and hand-washed with cold water. To remove the cover, unzip and remove foam core. Hand-wash with cold water and air dry ONLY. Daily maintenance can be done easily using a moist sponge or towel.
- 2 To clean the foam core remove the cover and spot clean with soap and water using a moist cloth. After cleaning, allow the foam to dry completely before replacing the cover and using.

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA foam cushions are guaranteed to be free from manufacturing defects. NOVA will repair or replace the identified defect for a period of up to 6 months of use. In the unlikely event of a defect please call our customer service line listed below for instructions. Items not covered include normal wear and tear, foam or fabric discoloration, flattening, density variation, improper use, rental use, and neglect. Damage due to natural acts is not covered.

**IMPORTANT WARNING**  
**READ THESE DIRECTIONS CAREFULLY BEFORE USE.**  
**FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

- Be sure cushion is in place and secure before each use.

Questions?

Please contact us at  
1-800-557-6682

info@novamedicalproducts.com  
novamedicalproducts.com

Thank you for choosing NOVA!

### Get Social with NOVA!

