User Guide for Nova 2611BK/2612BK Foam Wheelchair Back Cushions with lumbar support **Guía Sdel usuario para cojines de espuma & Espuma gelatinosa para 2611BK/2612BK Easy Rider**

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Placement on Wheelchair

Position the cushion on the back of the wheelchair so that the protruding lumbar support is closest to the seat and facing toward the user. Un-buckle the straps and place over the top and bottom of the back and securely fasten.

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To clean the Cover

The cover can be easily removed and machine-washed in cold water. To remove the cover pull apart the Velcro strip on the top of the cushion. Remove the support board from the back of the cushion. Fold the cushion and peel the cover back and off of the cushion. Machine-wash in cold water and air dry ONLY. Daily maintenance can be done easily using a moist sponge or towel.

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To clean the foam core

Remove the cover and spot clean the foam core or wipe down with a warm moist cloth. After cleaning allow the foam to dry completely before replacing the cover and using.

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Limited Warranty Information

Nova wheelchair cushions are guaranteed to be free from manufacturing defects. Nova will repair or replace the identified defect for a period of up to 12 months of use. In the unlikely event of a defect please call our customer service line listed below for instructions. Items not covered include normal wear and tear, foam or fabric discoloration, flattening, density variation, improper use, rental use, and neglect. Damage due to natural acts is not covered

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Warnings

Always consult a physician or rehabilitation specialist to for help choosing the proper cushion

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