


Relieves pain & pressure

Easy to inflate. Ready in minutes!

Item # 2619A-R Capacity not to exceed 225 lb

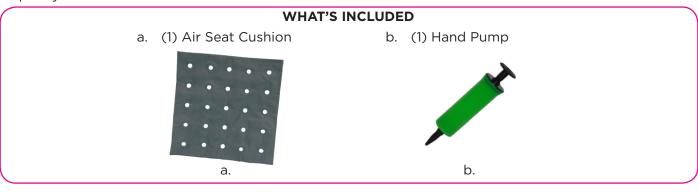



INSTRUCTIONS

Figure 1

To inflate:

Use the manual pump provided to inflate the Air Seat Cushion at the rubber valve (**Figure 1**).

To deflate:

Open and gently squeeze the base of the valve while applying pressure to the cushion.

IMPORTANT NOTES

- DO NOT overinflate
- The Air Seat Cushion will only appear semi-full to achieve its full benefit
- Cushion should be placed with the valve face down and to the rear
- · Reinflating your cushion may require less air

WARNING

Read all instructions carefully before use. Failure to do so may result in serious injury. NOVA Medical Products shall have no liability for any damage or injury resulting from improper use of this product.

CARE AND MAINTENANCE

Clean product regularly with soapy water. DO NOT wash or dry in laundry machines. This product should be at normal room temperature before inflating. Altitude and temperature may affect inflation.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Cushion and Pump

One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use soapy water, rinse and dry completely. Abrasive cloths may damage the surface. Regularly

......

inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US! We're here to help



✓ info@novajoy.com



1800 557 6682

novajoy.com



