

HOW TO USE YOUR



Easy Air Adjustable Toilet Seat Riser

Capacity not to exceed 250 lb
Item Number: 2628A-R




CUSTOM FIT

VARIABLE PRESSURE RELIEF

INSTRUCTIONS FOR YOUR EASY AIR ADJUSTABLE TOILET SEAT RISER

- 1** Secure toilet seat riser to toilet seat with velcro straps.
- 2** To release air and custom fit the cushion:
 - a.** Turn the valve clockwise (toward the front) until it is in the fully closed position.
 - b.** Sit on the seat and slowly turn the valve counter clockwise (toward the back) to release air from the cushion.
 - c.** Close the valve when you reach your desired comfort level.
- 3** To inflate the cushion or re-adjust it:
 - a.** Remove all weight from the cushion and open the valve by turning counter clockwise (toward the back).
 - b.** Allow the cushion to re-inflate and then close the valve by turning clockwise (toward the front).
 - c.** Follow instructions above to release air and custom fit.

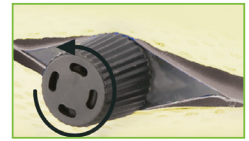

Comfort adjustment valve

CARE INSTRUCTIONS FOR YOUR EASY AIR ADJUSTABLE TOILET SEAT RISER

- 1** The cover can be easily removed and hand washed with cold water. AIR DRY only. To remove the cover, unzip and remove core. Daily maintenance can be done easily using a moist sponge or towel.
- 2** To clean the core remove the cover and clean with soap and water using a moist cloth. After cleaning, allow bladder to AIR DRY completely before replacing the cover and using.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA cushions are guaranteed to be free from manufacturing defects. NOVA will repair or replace the identified defect for a period of up to 6 months of use. In the unlikely event of a defect please call our customer service line listed below for instructions. Items not covered include normal wear and tear, foam or fabric discoloration, flattening, density variation, improper use, rental use, and neglect. Damage due to natural acts is not covered.

IMPORTANT WARNING
READ THESE DIRECTIONS CAREFULLY BEFORE USE.
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- Be sure cushion is in place and secure before each use.

Questions?
Please contact us at
1-800-557-6682
info@novamedicalproducts.com
novamedicalproducts.com
Thank you for choosing NOVA!

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Connect with us!

Facebook: novamedicalprod
Twitter: novamedicalprod
Twitter: novawalkertalk
Pinterest: novamedicalprod