

HOW TO USE YOUR



Bed Wedge with Half Roll Pillow

Item Number: 2699-R



EASES RESPIRATORY PROBLEMS

REDUCES STRESS ON SPINE

USING BED WEDGE AS A STANDARD WEDGE:

- 1** To sit up and use as a backrest, lay the wedge with the widest end down, narrowest end up & flat side against the wall/headboard.
- 2** To elevate your head and upper body, lay the wedge with the widest end against the wall/headboard, narrowest end under your body & flat side against the bed.



USING BED WEDGE WITH HALF ROLL PILLOW:

- 1** To elevate your head and upper body, lay the wedge with the widest end against the wall/headboard, narrowest end under your body & flat side against the bed.
- 2** Take Half Roll Pillow out of cutout and place on top of the wedge incline, with the flat side face down. If sleeping on your side, rest arm in the cutout.



Used as a standard bed wedge



Used with Semi Roll Pillow

CARE INSTRUCTIONS FOR YOUR BED WEDGE

- 1** The cover can be easily removed and hand-washed with cold water. To remove the cover, unzip and remove foam core. Hand-wash with cold water and air dry ONLY. Daily maintenance can be done easily using a moist sponge or towel.
- 2** To clean the foam core remove the cover and spot clean with soap and water using a moist cloth. After cleaning, allow the foam to dry completely before replacing the cover and using.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA foam cushions are guaranteed to be free from manufacturing defects. NOVA will repair or replace the identified defect for a period of up to 6 months of use. In the unlikely event of a defect please call our customer service line listed below for instructions. Items not covered include normal wear and tear, foam or fabric discoloration, flattening, density variation, improper use, rental use, and neglect. Damage due to natural acts is not covered.

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!

Facebook: novamedicalprod
Twitter: novamedicalprod
Twitter: novawalkerstalk
Pinterest: novamedicalprod