

HOW TO USE YOUR



FOLDING SEAT CANE

Item # 3090

Capacity not to exceed 175lb



No Tools
Required

1 For Walking:

- Folding Seat Cane must be in a straight and folded position when used as a walking cane.
- Make sure the center metal bar is locked into the seat clip located at the back of the seat. There should be a click to signal that the center metal bar is fully engaged (see Figure 1).



2 For Sitting:

- Unfold the Seat Cane so the seat comes outward in a flat position (see Figure 2). All 3 legs touch the ground evenly.
- Make sure the seat is secure in the flat position (see Figure 3).
- Place Folding Seat Cane on a flat and level surface (see Figure 4).

✓ Correct Sitting

Sit on the seat with the handle in front (see Figure 5).
Hold on to the handle for additional stability.

✗ Incorrect Sitting

DO NOT sit on the seat with the handle behind you or against your back (see Figure 6).



Correct Sitting



Figure 4

Incorrect Sitting



Figure 6

WARNING
Failure to use the cane with
correct seating position can
cause serious injury

WARNING

Read all instructions carefully. Failure to do so may result in serious injury.

- Check regularly to ensure that the Folding Seat Cane folds and unfolds properly
- When attempting to sit, make sure the seat is completely flat, and the cane holds securely in the unfolded position.



CARE AND MAINTENANCE

Regular cleaning is recommended. Use a non abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame

Non-Metal/Metal parts

One Year Limited Warranty

(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.



CARE • QUALITY • INNOVATION



THE NOVA PROMISE

SINCE 1993

NOVA believes in the power of joy. We are guided by joy, and inspired by our customers who *choose joy* everyday and every step of the way.

novajoy.com

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

Connect with us!

novajoy.com

 [novamedicalprod](#)

 [novamedicalprod](#)

 [novawalkerstalk](#)

 NOVA Medical Products