



How To Adjust Your Cane

1. Depress the brass knob push button on legs for vertical height adjustment
2. Lengthen or shorten **ALL 3 LEGS** of cane to the same level
3. Ensure brass knob is fully engaged to maintain desired height

How To Use

For Walking

1. Folding Seat Cane must be in a straight and folded position when using as a walking cane
2. Make sure the center metal bar is locked into the clip located at the back of the seat
3. Use the cane in the hand of the opposite side of your injured or weaker leg

For Sitting

1. Unfold Seat Cane so the seat becomes parallel to the floor
2. Make sure the seat is secure in the flat position
3. Place Folding Seat Cane on a flat and level surface
4. Sit on the seat with the handle in front (**Figure 1**)
5. Hold the handle for additional stability
6. **DO NOT** sit on the seat with the handle behind you or against your back. Doing so may result in injury or death

Figure 1



● **Correct for use**

CARE AND MAINTENANCE

Check regularly to ensure that the Folding Seat Cane folds and unfolds properly. When attempting to sit, make sure the seat is completely flat, all legs are adjusted to the same length, and the cane holds securely in the unfolded position.

⚠️ WARNING

Read all instructions carefully and consult with a qualified professional before using to avoid possible injury. **NOVA shall have no responsibility for failures resulting from improper use.** Do not misuse the product.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Connect with us!
novajoy.com
facebook: [novamedicalprod](#)
twitter: [novamedicalprod](#)
twitter: [novawalkerstalk](#)
YouTube: [NOVA Medical Products](#)