

HOW TO
ASSEMBLE YOUR



Toilet Safety Rails

Item # 8200, 8200-R, 8200-S
Capacity not to exceed 250 lbs.



QUICK ASSEMBLY

**Follow these
SIMPLE STEPS**




Adjustment Legs



Bracket



Arm Rest Frames

- 1** Remove all parts from box:
Two arm rest frames, 2 adjustment legs & 1 bracket.
 - 2** Remove the existing toilet seat and lid from toilet by unscrewing the screws located at the rear of the seat.
 - 3** Place the toilet safety frame bracket between the toilet opening and the tank with the rounded part of the bracket closest to the tank. The holes on the bracket should line up with the mounting holes on the toilet.
- 
- 4** Replace the toilet seat and lid and reinsert screws through the bracket and then through the toilet.
 - 5** Attach adjustment legs to each arm frame. Push in on brass pin and slide the adjustment leg onto the frame until the brass pin engages into the first hole.
 - 6** Place the frames next to the toilet and turn the small “L” shaped tube towards the toilet until the brass pins engage into the arms.
 - 7** Remove star knobs & insert side frames into the bracket. Adjust to the desired width and reinsert star knobs to lock side frames in place.
 - 8** Adjust the height of the legs by depressing the push-button and sliding the leg up or down to the desired height. Make sure both legs are fully engaged and adjusted to equal heights. (Arm rests should be parallel to the ground)

**HOW TO USE YOUR
Toilet Safety Rails**

- 1** When getting on and off the toilet use the toilet safety rails for extra stability. Make sure to not apply all of your body weight, as this product is for assistance only.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. please regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING
READ THESE DIRECTIONS CAREFULLY BEFORE USE.
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

Questions?
Please contact us at
1-800-557-6682
info@novamedicalproducts.com

Thank you for choosing NOVA!

Connect with us!
novamedicalproducts.com
facebook: novamedicalprod
twitter: novamedicalprod
twitter: novawalkerstalk
pinterest: novamedicalprod