

HOW TO ASSEMBLE & INSTALL YOUR


nova

TOILET SUPPORT RAILS

Item Number: 8201-R

Capacity not to exceed 300 lb

Easy assembly

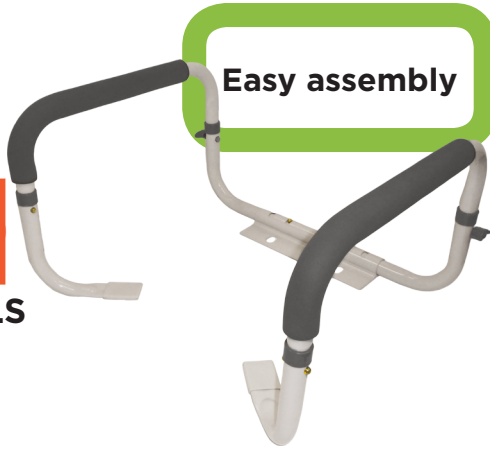

WHAT'S IN THE BOX

- a. (2) Armrests
- b. (2) Securing Screws
- c. (1) Bracket
- d. (1) Left Front Leg
- e. (1) Right Front Leg
- f. (2) Rear Frame Supports




⚠ WARNING

Read all instructions carefully and consult with a qualified professional before installing Toilet Safety Frame to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper installations or misuse of this product.**


ARM COMPONENT ASSEMBLY

Assemble arm components of Toilet Safety Frame as shown below:

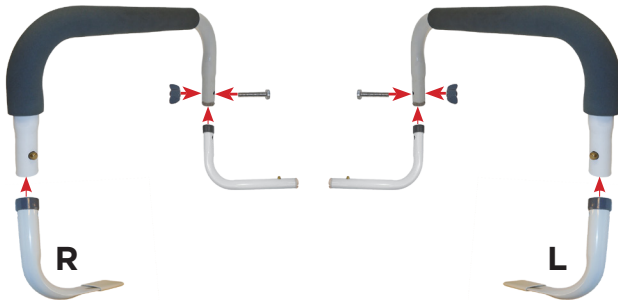

INSTALLATION

1

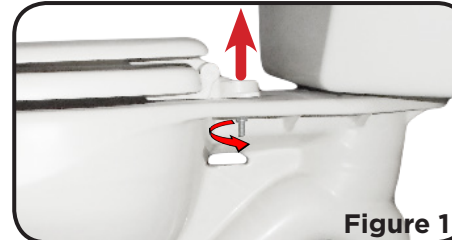


Take off existing toilet seat and lid by removing the screws in the rear (**Figure 1**).

2



Place mounting bracket on surface of toilet bowl and align the holes (**Figure 2**). The bracket tube should be closest to the tank.

3

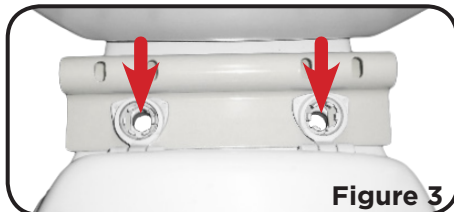

Replace toilet seat, and insert original bolts through mounting bracket holes and into toilet seat mounting holes (**Figure 3**). Use original nuts to tighten.

4



Depress push buttons and install arms into bracket (**Figure 4**). Adjust the Toilet Seat Rail arm width by engaging push buttons in desired holes of mounting bracket tube.

5



Ensure the Toilet Safety Frame's rubber sleeves rest securely on the front of the toilet rim under toilet seat and lid (**Figure 5**).

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- Device intended to be used as safety frame ONLY
- Product provides support, increases stability, and assists the end user
- Product is NOT designed to support the full weight of an individual
- Regularly check frame screws to ensure they are securely attached. Tighten if necessary

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Connect with us!
novajoy.com
facebook: novamedicalprod
twitter: novamedicalprod
twitter: novawalkerstalk
YouTube: NOVA Medical Products