

HOW TO USE YOUR



ADJUSTABLE BATH STOOL

Item # 9005

Weight capacity not to exceed 300 lb



**NO tools required
Quick installation**


**Easy to follow
step-by-step
instructions**

HOW TO ADJUST LEGS AND SEAT HEIGHT

Place the seat face down on a level surface with the legs of the stool facing up. Depress the brass push button on the legs and slide each leg to a desired height until a “click” sound is produced (**Figure 1**).

Ensure that each leg is set to the same height, is stable, and that all push buttons are fully engaged and locked before using.

Figure 1

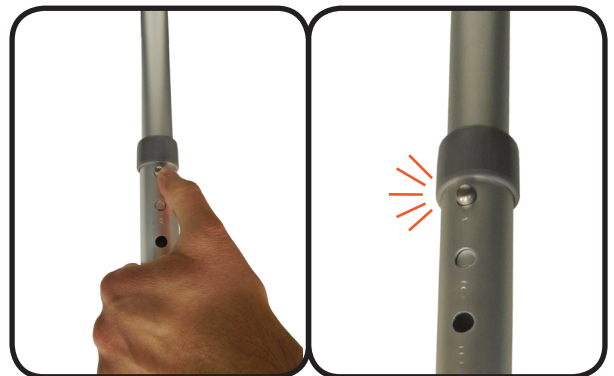

HOW TO USE

Bath and shower chairs are used by individuals who have difficulty or feel unsafe sitting in the bathtub or standing in the shower.

Place the bath stool in the shower or bath tub. Ensure that the stool is stable and ensure that all four legs make contact with the floor. Carefully sit down on the stool and apply even pressure around the entire seat.

CARE AND MAINTENANCE

Use non-abrasive cleaner or detergent, rinse with warm water. Abrasive cloths may damage the surface. Regularly inspect the bath bench and its parts for any corrosion, breakage, compression or wear. Regularly check the legs and ensure that they are securely set.

WARNING

Read all instructions carefully and consult with a qualified professional before adjusting to avoid possible injury. **NOVA shall have no responsibility for failures resulting from improper assembly or use.** DO NOT misuse the product.

- **DO NOT** use stool as an assistive device in entering and/or exiting a bathtub or shower
- **ONLY** use the bath stool in bathtubs and showers that are wide enough to ensure stability in all four legs
- Supervision or assistance while using this product is recommended for individuals with limited mobility

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame - Limited Lifetime Warranty

Non-Metal/Metal parts - One Year Limited Warranty
(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- All 4 legs must be adjusted to the same height (number) so that the stool sits level
- Make sure the legs are secure (brass button locked completely into hole) before using
- Check unit before each use
- ALWAYS use on a level surface
- DO NOT stand on the Bath Stool

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!
novajoy.com
facebook: [novamedicalprod](#)
twitter: [novamedicalprod](#)
twitter: [novawalkerstalk](#)
YouTube: [NOVA Medical Products](#)