

## HOW TO ASSEMBLE YOUR



### Deluxe Bath Seat with Arms and Removable Back

Item # 9036/9036-R  
Capacity not to exceed 300




No tools required  
Quick Assembly

Easy to follow  
step by step  
instructions

**1** Remove all parts from box:  
(Be sure all parts are included before assembling)

- a. (1) Seat
- b. (1) Backrest
- c. (2) Arms
- d. (4) Legs
- e. (2) L-shaped back tubes

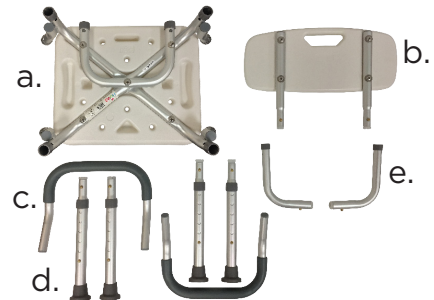


**2** Place the seat face down on a level surface. Insert the four legs into the frame. Depress the brass button and push the tube in until the button “clicks” and locks into position.



**3** Insert the L-shaped back tubes into the backrest tubes until the brass button “clicks” into locked position.



**4** Turn the chair on its side so that the tubing holes are facing upward. Insert backrest into tubing holes until the brass buttons “click” and lock into position.

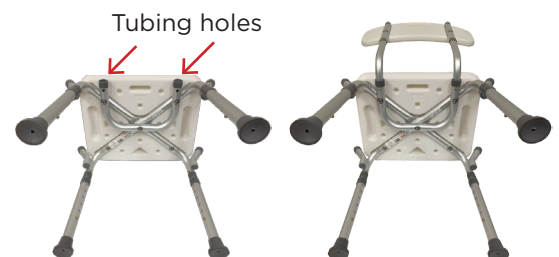

**5** Place the chair upright so that it is resting evenly on all four legs and insert the arms into the frame. Depress both brass buttons and push down until the brass buttons “click” and lock into position. Repeat this step for the opposite arm. (Note: Arms should curve inwards towards each other)



**6** Use the push buttons and numbered holes to adjust the stool to the desired seat height. Be sure the Bath Seat is level, stable and all push buttons fully engaged before using.

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

### IMPORTANT WARNING

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

- Chair legs must be adjusted to same height (number) so the bath seat sits level
- Make sure legs and backrest are secure (brass button locked completely into hole) before using
- Check before each use
- Use on a level surface
- Do not stand on Bath Seat

Questions?  
Please contact us at  
**1-800-557-6682**  
[info@novamedicalproducts.com](mailto:info@novamedicalproducts.com)  
Thank you for choosing NOVA!

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!  
[novamedicalproducts.com](http://novamedicalproducts.com)  
facebook: [novamedicalprod](#)  
twitter: [novawalkerstalk](#)  
YouTube: [NOVA Medical Products](#)