# **HOW TO** ASSEMBLE YOUR



Item # 9131/9131-R Capacity not to exceed 500 lb



NO tools required. Quick installation.

> Easy to follow step-by-step instructions

#### WHAT'S IN THE BOX

- Seat a. (1)
- (4) Adjustable legs
- b. a.

- c. (1)
  - **Backrest**
- d. (2) L-shaped backrest tubes
- e. (4)Knobs with washers




Place the seat face down on a level surface. Insert legs into the four corners of the seat. Depress the push button at the tip of the leg, and push the leg into the seat until the push button protrudes out of the hole. The push button should snap and lock the leg into position.

> Adjust all legs to the same desired height by depressing the push button and sliding the leg up or down. Slightly twist each leg to ensure push buttons are fully engaged and stable.




Turn the seat on its side so that the backrest cutouts face upward. Place backrest tube against the bottom of the seat and align the tubing and screw holes. The end of the backrest tube with the push button will point away from the legs.

> Insert knobs with washers into the screw holes. Tighten knobs to secure backrest tubes to the seat. Repeat for the other side.

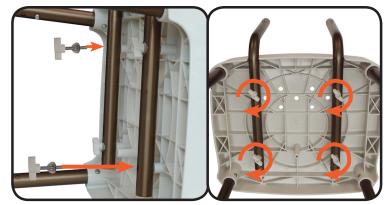

Place bath seat on its legs and install the backrest onto the backrest tubes. Depress push buttons on the backrest tubes and slide the backrest down until it snaps into place and is secure. Ensure all parts are secure before using bath seat.

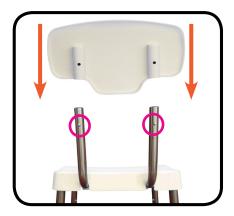



Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.

- All legs MUST be adjusted to the same height to ensure level seating and safety
- Use EXTREME caution on wet tub or floor surfaces
- DO NOT stand on bath seat
- DO NOT use if bath seat is damaged. Check frequently for damage or looseness.
- This product is rated for persons weighing up to **500 lb** when properly assembled

## **HOW TO USE**

Place the bath bench in the shower or bath tub. Ensure that the bench is stable and not wobbly. Slowly lower yourself onto the bath seat before turning on shower.

# CARE AND MAINTENANCE

Check regularly to make sure all legs on the bath seat are securely locked. Use a mild detergent and warm water to clean.

### WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame Limited Lifetime Warranty
Non-Metal/Metal parts One Year Limited Warranty
(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

#### **IMPORTANT WARNING**

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US! We're here to help



CARE • QUALITY • INNOVATION

THE **NOVA** PROMISE

Connect with us! novajoy.com facebook: novamedicalprod twitter: novamedicalprod twitter: novawalkerstalk YouTube: NOVA Medical Products