

HOW TO USE YOUR



32" Folding Reacher with Suction Cup Tips

Item # FR-32



HELPFUL
EVERYDAY

Simple to use and
easy to clean

To Unfold:


Swivel arm into extended position and push the blue safety lock to engage in the extended position. Be sure arm is in the locked position before using.

To Fold:

Push the blue safety lock to unlock the reacher from the extended position and swivel the arm back.

To Lock The Trigger:

Pull the squeeze handle and push the sliding trigger back

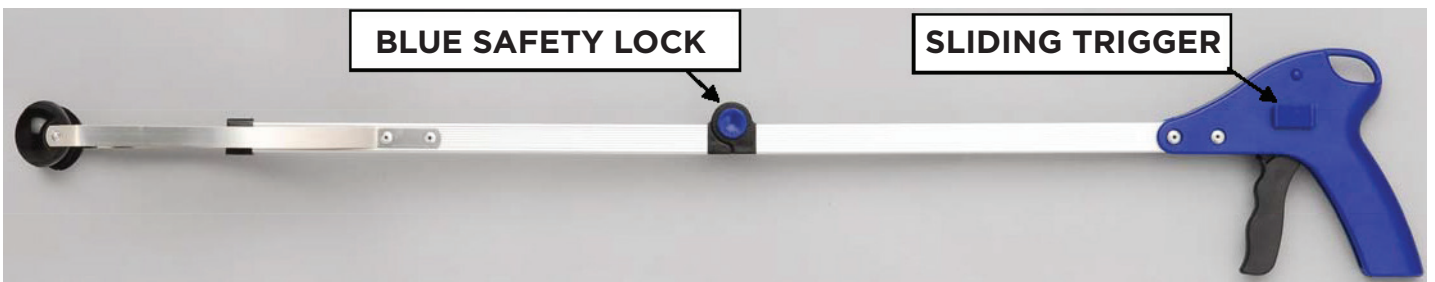

Para Desplegar

Gire el brazo en posición extendida y empuje el seguro de bloqueo azul para engranar la posición extendida. Asegúrese que el brazo este en posición de cierre antes de utilizar.

Para Doblar

Empuje el seguro de bloqueo azul para desbloquear el alcanzador de posición extendida y gire del brazo

Para bloquear el gatillo

Tire de la palanca de contracción y empuje el gatillo deslizante hacia atrás.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

**READ THESE DIRECTIONS CAREFULLY BEFORE USE.
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

- Be sure reacher is locked in place and secure before each use
- Stand with feet flat on floor when using the reacher

Questions?
Please contact us at
1-800-557-6682
info@novamedicalproducts.com

Thank you for choosing NOVA!

Connect with us!
novamedicalproducts.com
facebook: novamedicalprod
twitter: novamedicalprod
twitter: novawalkerstalk
pinterest: novamedicalprod