

PLEASE READ BEFORE SEARCHING FOR THE PART NEEDED



Many of the parts pictured on this breakdown may look identical to the part or parts needed but the measurements may be different. Please check the serial number on your rolling walker and make sure it corresponds with the serial number on this part diagram highlighted in yellow. The serial number on the 4200C rolling walkers are located on a sticker under the seat on the crossbar the seat is attached to. Refer to the picture below. If you cannot find the serial number, please call customer service for assistance at (800) 557-6682 option 1.

Thank you



ITEM NUMBER 4200C CRUISER CLASSIC


SERIAL NUMBER BEGINNING WITH "NV"



40023BK



C4203CB

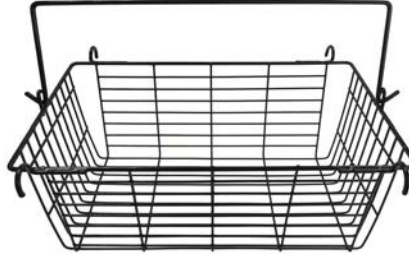

H-1002




P40000



P40003



P40006

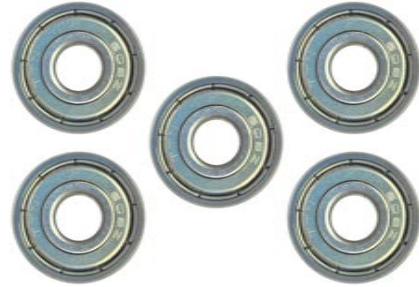

P40009



P40010



P40012



P40015



P40016



P40028



P40035



P200WA



P42006



P42007



P42008



P42009



P42010



P42018



P42019



P42022



P42029




P42034



P42036



P42037

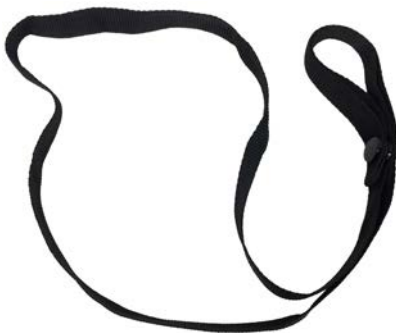

P42038



P42046



P43003

P43004

V42005

V42006



V42007

V42064

V42065



V42066

V42076

V42077



V42078

V42089

V42093



V42094




VT-33207



CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

ITEM NUMBER 4200C	CRUISER CLASSIC	
	SERIAL NUMBER BEGINNING WITH "NV"	
ITEM/PART NUMBER	DESCRIPTION	
40023BK	RUBBER TIPS BLACK	PAIR
C4203CB	BASKET	
H-1002	LOCK N LOAD	
P40000	COVER FOR REAR AXLE NUT	
P40003	SILENCER	
P40006	BEARINGS FOR WHEELS	SET OF 5
P40009	COVER FOR FORK BEARING	
P40010	BUSHING BEARING FOR FORK	
P40012	WASHER FLAT STEEL	
P40015	COVER FOR FRONT AXLE NUT	
P40016	NUT FOR FRONT AXLE	SET OF 5
P40028	NUT FOR REAR AXLE & FOLDING MECHANISM	
P40035	SCREW STEM FOR FRONT FORK	
P4200WA	WHEEL ASSEMBLY REAR COMPLETE	SET
P42006	CROSSBAR FOR FOLDING MECHANISM	
P42007	SCREW FOR FOLDING MECHANISM & A-FRAM PLUG	
P42008	WASHER CONTOURED STEEL	
P42009	SPACER FOR A-FRAME PLUG	
P42010	SPACER CONTOURED PLASTIC	
P42018	SPRING FOR REAR WHEELS	
P42019	WHEEL 5"	
P42022	FORK FRONT SWIVEL (INCLUDES COVER AND BUSHING)	SET
P42029	SPACER FOR REAR WHEEL & REAR AXLE	PAIR
P42034	AXLE REAR	
P42036	NUT FOR FOLDING MECHANISM CROSSBAR	
P42037	STRAP FOR FOLDING MECHANISM	
P42038	FOLDING MECHANISM (INCLUDES HARDWARE)	SET
P42046	AXLE FRONT	
P43003	HAND GRIP LEFT	
P43004	HAND GRIP RIGHT	
V42005	HANDLE BAR LEFT (INCLUDES HAND GRIP)	SET
V42006	HANDLE BAR RIGHT (INCLUDES HAND GRIP)	SET
V42007	CLAMP FOR SEAT	
V42064	SCREW, SPACER & NUT FOR A-FRAME PLUG & BACKREST RECEIVER	SET
V42065	BACKREST BLUE	
V42066	BACKREST RED	
V42076	RECEIVER FOR BACKREST	

V42077S	SCREWS FOR SEAT	SET
V42078	SEAT PADDED	
V42089	KNOB FOR HANDLE HEIGHT ADJUSTMENT	
V42093	CUSHION FOR BACKREST	
V42094	PLUG FOR A-FRAME	
VT-33207	BEARING FOR FORK	

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682